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Softform Heelpad

Instructions for Use

- Please read the instructions carefully before use.
- Remove all packaging before use.
- Position under heel/heels so that the heel/heels immerse into the gel sac/sacs.
- The Softform Heelpad is designed to be used in conjunction with the Softform Odstock Wedge to provide improved support of the lower limb/limbs.

Product Description

The Invacare range of Softform Heelpads offers exceptional pressure reducing qualities, specific to the individual needs of the body.

The Softform Heelpad is designed to reduce pressure on the vulnerable heel area. The controlled volume of gel within the sac/sacs allows pressure displacement and even weight distribution. Pressure on the patient's heel/heels is kept to a minimum when the heel/heels is/are in contact with the gel sac/sacs.

The heavy duty water-resistant cover provides a vapour-permeable, two-way stretch surface to promote patient comfort and maximise the effectiveness of the foam core.

Softform Heelpad	Length 68cm	Width 48cm	Height 3.5cm
------------------	----------------	---------------	-----------------

If you require more information on the Softform accessory range please telephone the customer services department at Invacare on +44 (0) 1656 776222 who will be happy to help you.

Repositioning

It is very important for the patient to reposition themselves, or to be repositioned, on a regular basis. This relieves pressure; helps prevent both tissue compression and potential ulcer formation.

Cleaning, Disinfection and Re-processing

- The cleaning protocols should be undertaken as part of a cleaning regime or in the event of a change of user. The cover can be wiped down using a solution of neutral detergent and hot water, applied with a single-use wipe. Cleaning agents containing phenols or alcohols should not be used.
- In the event of the cover becoming contaminated, please ask your Infection Control Officer for advice or alternatively to disinfect the cover apply a 0.1% chloride solution with a single use wipe and rinse with clean water.
- The Heelpad is provided with disposable Platilon covers, facilitating improved infection control. Additional covers may be purchased separately.

Changing the cover

The cover cannot be changed as it is welded to prevent fluid ingress. Disposable covers are provided to facilitate infection control.

Caution

Minimum user weight: 25Kg

Maximum user weight: 249Kg

Quality and Flame Retardancy

Quality is fundamental to the company's operation, working within the disciplines of ISO 9001:2000 and ISO 13485:2003. All Invacare products feature the CE mark, in compliance with the Medical Devices Directive.

The foam used to manufacture the Invacare Sofform Heelpad is independently tested and certified in accordance with BS 5852:2006 ignition sources 0, 1 and crib 5. The Cover is tested to BS7175:1989

Manufactured by Invacare Ltd
Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ UK
Tel: +44 (0) 1656 776222 Fax: +44 (0) 1656 776220

In keeping with our policy of continuous improvement, we reserve the right to modify designs without prior notice.
Sofform is a registered trademark of Invacare LTD.

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Softform is a registered trademark of Invacare LTD.
Hersteller: **INVACARE UK OPS LTD.**, Pencoed Technology Park, Pencoed,