

### Simple adaptions that make a difference



## **Adjustments and accessories**

Our Ocean Ergo range offers an advanced solution in shower chairs. Each chair is designed to offer high levels of adjustments and configurability as standard. For further customisation, our latest range of accessories provides solutions for extra comfort, support and safety.



#### **Footplates**

- Angle adjustable, flip-up and textured footplates with heel loops
- > No tools required

#### **Footrests**

- Height adjustable, swing away and detachable footrests
- > No tools required

#### **Armrests**

 Ergonomic, swing away armrests can be adjusted to different heights

#### Seat height

- > Seat to ground height can be adjusted between 440-600 mm\* or 475-600 mm\*\*
  - \*Ocean Ergo, Ergo XL and 24" Ergo models \*\*Ocean 24" Ergo XL and VIP models
- > No tools required

## **Accessories**

#### Seating and backrests



#### Soft seat

Redesigned to mirror the contours of the new ergonomic seat plate, the softer feel gives users with sensory or skin protection issues added relief. Smaller aperture also available for shorter or thinner users. Easy to fit and clean.



#### Soft seat insert

Available as a standard version or smaller version. Can be used with the soft seat when the commode function is not required.



#### **Ergonomic soft seat**

Manufactured from viscoelastic foam, it offers a much softer feel for those with skin integrity or sensory issues. The foam creates greater depth of immersion promoting increased pelvic stability and positioning.



#### Variable soft seat

Hygiene cut out can be positioned in all four directions, improving access for carers assisting with personal hygiene.



#### **Universal soft seat**

Comfortable, closed seat surface for when commode function is not in use.



#### **Solid backrest**

The solid back support offers an alternative to the standard tension adjustable backrest, when firmer support is required to achieve a more upright posture. Easy to clean.



#### **Armrest pads**

New design offers improved support and comfort for the user (not for XL armrests).



#### XL back

Increases the width between the armrests by 80 mm. Can be retrofitted to standard Ocean Ergo models.



#### **Wider XL armrests**

Increases the distance between the armrests by 40 mm on each side.



#### **Soft backrest cushion**

For increased comfort. Can also be used to reduce the overall seat width and depth, helping to position smaller users.



#### **Comfort backrest**

An alternative to the standard backrest with a softer feel material for increased comfort.



#### Safety



**Safety bar**For additional safety and reassurance.



Safety lock for armrest Locks armrest into the vertical or horizontal position.

**Positioning** 



anti-tippers
For additional safety
plus front and rear
stability.

Front and rear



Tilt-stop
An aid for those who are hoisted in/out of the chair, the tilt-stop screws ensure the frame remains horizontal.



Safety belts

Broad pad manufactured from soft material with single pull buckle. Can be used across the pelvis or in addition across the chest for positioning and safety. Easy to attach. Machine washable.



Lateral supports

Adjustable in height via three possible positions on the backrest cane. Broad lateral pads can be repositioned for depth adjustment, comfort and effective pressure redistribution.



**Ergonomic headrest** holder

Offers multidirectional adjustment to correct or accommodate for more complex head/neck positioning needs.

Pad not included.



Ergonomic headrest/ neckrest

Offers exceptional management of the head position.

Neckrest shown.



Calf and amputee support with foot support

Offers both height and angle adjustment with a contoured pad for added positioning and comfort. Including a foot support for those who may need to elevate their lower limb(s) for medical reasons or comfort.



Calf and amputee support

Excluding footrest.



Splash guard

Clips on easily for use with the standard plastic seat, standard soft seat and small soft seat.



**Toilet pan** 

Ergonomic handle allows the pan to be easily removed and replaced. Features an integrated lid holder, so the lid is not misplaced during use.

## Ocean Ergo

# Customise each of our **Ocean** Ergo chairs

Our Ocean Ergo range of shower chairs can all be customised to suit a wide number of needs. By making a simple adjustment or by adding one of our easy-to-fit accessories, each of our modular shower chairs are more versatile than ever before.



Ocean Ergo

An entry level attendant propelled model



Ocean Ergo XL

An entry level attendant propelled model with a heavier maximum user weight



Ocean 24" Ergo

An entry level self-propelling model



Ocean 24" Ergo XL

An entry level self-propelling model with a heavier maximum user weight



Ocean VIP Ergo

A tilt-in-space model for additional seating and positioning support



#### Ocean Dual VIP Ergo

A tilt-in-space model with recline for additional seating and positioning support

## Ocean Ergo

# **Compatibility**

- ✓ Compatible
- x Not compatible
- + Included

Most accessories a	re cross compatible
with all Ocean Ergo	o models.

Standard soft seat AP1603142 Soft seat insert AP1603143 Small soft seat AP1603145 Small soft seat insert AP1603152	\(  \)	Ergo /	<b>✓</b>		Ergo	Ergo
Soft seat insert AP1603143 Small soft seat AP1603145	1		✓	/		
Small soft seat AP1603145		1		<b>√</b>	✓	<b>✓</b>
	<b>√</b>		✓	<b>✓</b>	1	<b>√</b>
Small soft seat insert AP1603152		✓	✓	<b>✓</b>	<b>✓</b>	<b>✓</b>
	✓	✓	✓	✓	✓	<b>✓</b>
Splash guard AP1603155	✓	✓	✓	✓	✓	✓
Universal soft seat AP1603154	✓	✓	✓	✓	✓	✓
Ergonomic special soft seat AP60121311	✓	✓	✓	✓	✓	<b>✓</b>
Variable soft seat 1544469	✓	✓	✓	✓	✓	✓
Safety bar 2.0 AP60123732	✓	✓	x	x	✓	✓
Safety bar 2.0 XL (only with armrests 2.0) AP60123771	x	x	✓	✓	x	x
Safety bar XL (only for XL chairs with old armrests) 1470078	x	x	✓	✓	x	x
Chest belt 1470081	1	✓	✓	✓	✓	✓
Pelvic belt 1470082	✓	✓	✓	✓	✓	✓
XL back AP1604015	1	1	+	+	x	x
Solid backrest 1575299	1	✓	x	x	✓	<b>✓</b>
Soft backrest cushion 1558769	<b>✓</b>	1	/	<b>✓</b>	1	1
Armrest pad (1 pad; only for new armrests 2.0) AP1643748	<b>✓</b>	1	/	<b>✓</b>	1	<b>√</b>
Wider XL armrests 1470086	1	1	1	<b>√</b>	✓	1
Safety lock for XL armrest 1470077	<b>✓</b>	1	/	<b>✓</b>	1	<b>✓</b>
Safety lock 2.0 (for standard armrests 2.0) AP1643760	<b>✓</b>	1	/	<b>✓</b>	1	1
24" wheels (set) 16379	1	+	<b>✓</b>	+	x	x
Calf and amputee support right AP1643573	<b>✓</b>	1	/	<b>✓</b>	1	1
Calf and amputee support left AP1643574	1	1	<b>✓</b>	<b>√</b>	1	<b>√</b>
Calf support with foot support right AP1643543	/	<b>√</b>	<b>✓</b>	<b>√</b>	<b>√</b>	<b>√</b>
Calf support with foot support left AP1643572	/	1	/	<b>√</b>	1	<b>√</b>
Lateral support 1535077	1	1	<b>✓</b>	<b>√</b>	1	<b>√</b>
Ergonomic headrest holder AP1531613	1	1	/	<b>√</b>	1	<b>√</b>
Ergonomic headrest 1531465	<b>√</b>	1	<b>√</b>	<b>√</b>	<b>√</b>	<b>√</b>
Ergonomic neckrest 1531608	<b>√</b>	1	1	<b>√</b>	1	<b>√</b>
Toilet pan 10230	1	1	/	<b>√</b>	1	<b>√</b>
Toilet pan holder AP1602078	<b>√</b>	1	<b>✓</b>	<b>√</b>	1	<b>√</b>
Front and back anti-tipper Ergo VIP models AP1603992	x	х	x	x	1	<b>√</b>
Front anti-tipper Ocean Ergo models AP1601986	1	1	1	/	x	х
Comfort soft backrest VIP Ergo models AP1603606	x	х	x	х	<b>√</b>	<b>√</b>
Comfort soft backrest Ocean Ergo models AP1604136	/	1	Х	Х	х	х
Comfort soft backrest Ocean Ergo XL models AP1604016	Х	х	<b>√</b>	<b>√</b>	х	х
Tilt-stop screw set AP60129156	Х	Х	Х	X	<b>✓</b>	<b>√</b>

T: +44 1656 776222 uk@invacare.com www.invacare.co.uk

www.invacare.ie

Invacare UK & Ireland Unit 4 Pencoed Technology Park Bridgend CF35 5AQ

