

Skirt for Aquatec Ocean

Intended use

This product has been designed to be used with the Invacare® Aquatec® range of Ocean shower commode chairs to provide a protective barrier between the commode aperture and automatic toilet systems, such as Clos-O-Mat and Gerberit.

Instructions for use

Remove the soft seat from the Ocean shower chair revealing the hard plastic seat. Place the fabric centrally on the plastic seat ensuring the protection skirt is inserted through the commode recess, and ensuring that the shorter side of the skirt is placed towards the back of the shower chair. This is necessary so that the fabric does not affect the function of the automatic toilet's shower arm. The soft seat **MUST** then be replaced to hold the fabric in place.



For effective use, ensure the flap sits neatly through the commode aperture and sits downwards into the toilet bowl.

Cleaning and care

The fabric can be wiped down using a solution of neutral detergent between uses. For heavy soiling the fabric can be removed and machine washed at a temperature no more than 71 degrees centigrade. Do not use phenols or alcohols to clean the fabric.

Quality

This product is provided as a protective skirt only and is designed to help reduce any splashing during cleaning and to assist the drying process by reducing the escape of warm air between the commode seat and the automatic toilet. This product is not a medical device.

Warranty

12 months.

Manufactured by:

Invacare Ltd, Unit 4, Pencoed Technology Park, Bridgend CF35 5AQ, Tel: 01656 77622

e-mail: ordersUK@invacare.com web: www.invacare.co.uk

In keeping with our policy of continuous improvement, we reserve the right to modify designs without prior notice.