


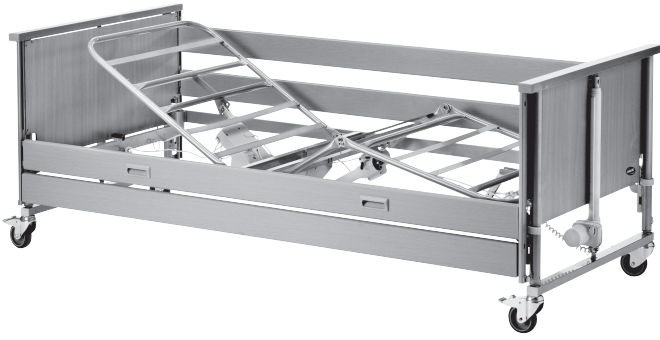

Invacare® Medley® Ergo

Medley Ergo, Medley Ergo Low

en Bed

User Manual

This manual **MUST** be given to the user of the product.
BEFORE using this product, this manual **MUST** be read and
saved for future reference.



Yes, you can.

Contents

1 General	3	4.6 Adjusting the leg section	17
1.1 Introduction	3	4.7 Emergency positions	17
1.2 Symbols in this Document	3	4.7.1 Emergency release of a mattress support section	17
1.3 Intended Use	3	4.7.2 Transportation and Storage	18
1.3.1 Intended Operator	3	5 Options and accessories	19
1.4 Service Life	3	5.1 List of available options	19
1.5 Compliance	4	5.2 Transport brackets	19
1.5.1 Product-specific standards	4	6 Maintenance	21
1.6 Warranty Information	4	6.1 General maintenance information	21
1.7 Limitation of Liability	4	6.2 Daily inspections	21
2 Safety	5	6.2.1 Daily Inspection Checklist	21
2.1 General safety information	5	6.3 General servicing information	21
2.2 Mattresses	6	6.3.1 Checklist maintenance	21
2.3 Safety information on electromagnetic interference	7	Checkpoints	21
2.4 Labels and symbols on the product	7	6.4 Inspection after relocation - Prepare for new user	22
2.4.1 Identification label	7	6.5 Cleaning and Disinfection	22
2.4.2 Other Labels	7	6.5.1 General Safety Information	22
Label on 3/4 length side rails	8	6.5.2 Cleaning Intervals	22
3 Setup	9	6.5.3 Cleaning by Hand	22
3.1 General Safety Information	9	6.5.4 Disinfection Instructions	22
3.2 Main parts of the bed	9	In Domestic Care	22
3.2.1 Other Items	9	In Institutional Care	22
3.3 Assembling the bed	9	6.6 Lubrication	22
3.3.1 Attaching Labels	10	7 After Use	23
3.4 Install Rastofix	10	7.1 Disposal	23
3.5 Control Box	10	7.2 Reconditioning	23
3.6 Wiring	11	8 Troubleshooting	23
3.7 Installing side rails	11	8.1 Troubleshooting electrical system	23
3.7.1 Scala 2 side rail	12	9 Technical Data	24
Mounting side rail	12	9.1 Materials	24
Dismounting side rail	12	9.2 Dimensions bed	24
3.7.2 Installing Aria and Bella side rail	12	9.3 Weights	24
Assembling the gliding system	12	9.4 Mattress dimensions	24
Attaching the gliding system and installing the side rail	12	9.5 Electrical data	25
3.8 Installing the mattress support extension	13	9.6 Environmental conditions	25
3.9 Dismantle the bed	14	10 Electromagnetic Compatibility	26
4 Operating the bed	15	10.1 General EMC information	26
4.1 General Safety Information	15	10.2 Electromagnetic emission	26
4.1.1 Low Heights	15	10.3 Electromagnetic Immunity	26
4.1.2 Mattress retainers	15	10.3.1 EMC test specifications	27
4.2 Hand control	15		
4.2.1 Locking function	15		
4.3 Castors and brakes	16		
4.3.1 Castor brake	16		
4.4 Operating side rails	16		
4.4.1 Operating Scala 2 side rail	16		
4.4.2 Aria and Bella side rail	16		
4.5 Lifting pole	17		
4.5.1 Place the lifting pole	17		
4.5.2 Adjusting the handle height	17		

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Only use this product if you have read and understood this manual. Seek additional advice from a healthcare professional who is familiar with your medical condition and clarify any questions regarding the correct use and necessary adjustment with the healthcare professional.

Note that there may be sections in this document, which are not relevant to your product, since this document applies to all available models (on the date of printing). If not otherwise stated, each section in this document refers to all models of the product.

The models and configurations available in your country can be found in the country-specific sales documents.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

For more information about the product, for example product safety notices and product recalls, contact your Invacare distributor. See addresses at the end of this document.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

1.2 Symbols in this Document

Symbols and signal words are used in this document and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.



WARNING!

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.



CAUTION!

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.



NOTICE!

Indicates a hazardous situation that could result in damage to property if it is not avoided.



Tips and Recommendations

Gives useful tips, recommendations, and information for efficient, trouble-free use.

Other Symbols

UKRP UKRP

Indicates if a product is not manufactured in the UK.

1.3 Intended Use

The medical bed is electrically operated, adjustable in height and features a profiling support surface intended to be used in combination with a mattress. Medical beds are designed for support of the patient and used for the mitigation, relief, and recovery from certain medical conditions. The medical bed also supports and eases the caregivers' working environment.

The Medical bed is intended to be used on:

- *Application environment 3*; Long-term care in a medical area where medical supervision is required and monitoring is provided if necessary and medical electrical equipment used in medical procedures may be provided to help maintain or improve the condition of the patient.
- *Application environment 4*; Care provided in a domestic area where medical electrical equipment is used to alleviate or compensate for an injury, disability or disease.

Intended Users

A person with a medical condition that needs to be cared for in bed.

Indications

The bed is intended for adult users, having a physical size equal to or more than 146 cm, a weight equal to or more than 40 kg and a body mass index (BMI) equal to or more than 17.

The bed is intended for indoor use only.

Maximum user weight is 145 kg and safe working load is 180 kg.

Contra-indications

The bed is not intended for transportation of users. It is mobile within a room with a user in it. Castors are lockable.

The bed is not intended for persons with a psychosocial disability.



WARNING!

Any other or incorrect use could lead to hazardous situations.

Invacare accepts no liability for any use, change or assembly of the product, other than stated in this user manual.

1.3.1 Intended Operator

A healthcare professional or private person who has received proper training is the intended operator of this product.

The hand control can also be operated by the occupant of the bed.

1.4 Service Life

The expected service life of this product is five years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

1.5 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I.

This product features the UKCA mark, in compliance with Part II UK MDR 2002 (as amended) Class I.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

We comply with the current environmental legislations WEEE and RoHS.

1.5.1 Product-specific standards

The product has been tested and conforms to EN 60601-2-52 (Particular requirements for the basic safety and essential performance of medical beds) and all related standards.

For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

1.6 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

1.7 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

2 Safety

2.1 General safety information

This section of the manual contains general safety information about your product. For specific safety information, refer to the appropriate section of the manual and procedures within that section.



WARNING!

Risk of injury or damage

- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, Invacare provider or qualified technician before attempting to use this product.
- Do not make any unauthorized alterations or modifications to the product.



WARNING!

Risk of injury or damage

Inappropriate handling of cables, incorrect connections and use of unauthorized equipment can cause electrical shock and product failure.

- Do not kink, shear or otherwise damage the cables of the product.
- Make sure that no cables are jammed or damaged, when the product is used.
- Disconnect the plug from the mains before moving the bed.
- Make sure the wiring is correct and connections are proper.
- Do not use unauthorized equipment.



WARNING!

Risk of tripping, entanglement or strangulation

Improper routing of cables may cause tripping, entanglement or strangulation hazard.

- Ensure all cables are routed and secured properly.
- Ensure there are no loops of excess cable extending away from the product.



WARNING!

Risk of entrapment / suffocation

There's a risk of entrapment / suffocation between mattress support, side rail and bed end or between moving parts and objects placed nearby the bed.

- The bed must not be used by persons under 12 years of age, or by persons with a body size equivalent to an average 12 years old or smaller.
- The bed, in combination with side rails must not be used by persons having a physical size less than 146 cm, a weight less than 40 kg or a body mass index (BMI) less than 17.
- Due to mattress compression, an increased risk may occur over time. Periodically monitor gaps between the bed, mattress and/or side rail. Replace mattress if the gaps may lead to entrapment.



WARNING!

Risk of injury or damage

Ignition sources can cause burns or fire.

- Bed must be placed with a safety space from possible ignition sources (heater, fireplace, etc.)
- The patient and assistants must not smoke while using or operating the bed.



WARNING!

Risk of injury or damage

To avoid injury or damage when operating the product:

- Close supervision is necessary when the product is used near children or pets.
- Do not let children play with the product.



WARNING!

Risk of injury or damage

Incorrect application and/or distribution of weight may result in injury or damage.

- Avoid loads at the extremities.
- Movable parts shall only be actuated after the user is properly and safely laid down.
- Do not exceed the safe working load, as the bed can get damaged or tip over.



WARNING!

Risk of slipping through the openings

The bed fulfills all requirements regarding maximum distances. However, it is possible that persons with small body dimensions slip through the openings between the side rails or through the opening between the side rail and the mattress support.

- Pay special attention, if the bed is used for the care of persons with small body dimensions.

**CAUTION!**

- For a person entering or exiting the bed, always lower the bed to an appropriate height. For entering or exiting the bed, the backrest section of the mattress support can be used to assist the positioning of a person's back in a more seated position. The backrest section of the mattress support is not intended to support and/or raise the full body of a person. Make sure the patient is laid down with his back at the centre of the backrest section. While the backrest section is being raised, the allowed maximum weight for the backrest section is 45% of the maximum safe working load.
- Always lower the bed to the lowest position before leaving a person in the bed unattended.
- Make sure that there is nothing under, over, or near the bed that can obstruct the height adjustment, like for example furniture, lifts or window frames.

**CAUTION!**

Non-original or wrong accessories and/or optional parts may affect the function and safety of the product.

- Only use original accessories and/or optional parts for the product in use.
- Due to regional differences, refer to your local Invacare catalog or website for available accessories and/or optional parts or contact your local Invacare representative. See addresses at the end of this document.

**CAUTION!****Risk of injury or damage**

The product can get hot when exposed to sunlight or other heat sources.

- Do not expose the product to direct sunlight for prolonged periods.
- Keep the product away from heat sources.

**CAUTION!**

There is a risk of entrapment of fingers in the bed moving parts.

- Pay attention to your fingers.

**NOTICE!**

The bed has no isolator (main switch). If the bed needs to be electrically disconnected, it has to be unplugged from the mains socket.

- Always place the bed in a way so the mains socket is easily accessible to unplug the bed.

**NOTICE!**

Accumulation of lint, dust and other dirt can impair the product.

- Keep the product clean.

2.2 Mattresses

**NOTICE!**

- A risk assessment is always to be carried out by a professional prior to the use of a mattress with an Invacare bed, in order to assess its compatibility and safety of use with the bed. Special attention must be paid prior to the use of a Dynamic mattress.
- If you have doubts regarding the use of your Invacare bed with a mattress, contact your Invacare provider for guidance.


**WARNING!****Safety aspects regarding combination of side rails and mattresses:**

To get the highest possible safety level, when using side rails on the bed, the minimum and maximum measures for mattresses, must be respected.

- For correct mattress measures see mattress table in chapter 9 *Technical Data*, page 24.



**WARNING!****Risk of entrapment and/or suffocation**

- The user could get trapped and/or suffocate, if the horizontal space, between the mattress side and the inside of the side rail, is too big. Follow the minimum width (and length) of mattresses in combination with a side rail, as stated in the mattress table in chapter 9 *Technical Data*, page 24.
- Be aware that using very thick or soft mattresses (low density), or a combination of these, increases the risk.

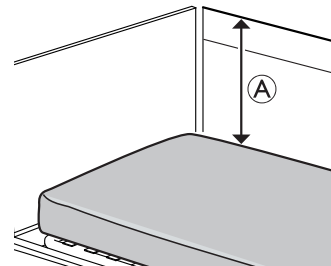
**WARNING!****Risk of falling**


The user can fall over the edge and get seriously injured, if the vertical distance ① between the top of the mattress and the edge of the side rail/bed end, is too short. See image above.

- Always keep a minimum distance ① of 22 cm.
- Follow the maximum mattress height in combination with the side rail as stated in the mattress table in chapter 9 *Technical Data*, page 24.

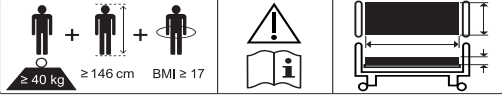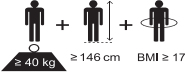	
	Definition of min. weight, min. height and min. body mass index of an adult user. See 1.3 <i>Intended Use</i> , page 3
	Refer to user documentation for the correct mattress measures. See 9 <i>Technical Data</i> , page 24

Label on 3/4 length side rails

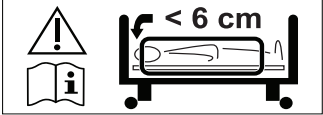
Indicates the maximum space between side rail and the head bed end. See the mounting instruction in this manual or the instruction delivered with the side rail.

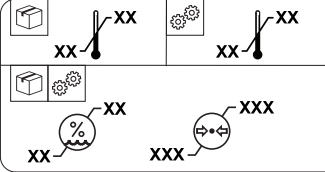	 = XXX
	Total weight of the product with the maximum safe working load applied
	Temperature limit
	Humidity limitation
	Atmospheric pressure limitation
	Transport and storage conditions
	Operation conditions


Indicates areas with increased risk of squeezing feet on the lower sides of the bed ends near the castors. See 4.1.1 <i>Low Heights</i> , page 15. (Only present on the low versions of this product)


Marks out the location of connection for means of potential equalization. (Not present on all versions of this product)

3 Setup

3.1 General Safety Information


WARNING!
Risk of injury or damage

Damaged parts may affect the safety of the product.

- Check all parts for shipping damage before use.
- In case of damage, do not use the product and contact your Invacare provider for further instructions.


WARNING!
Risk of injury or damage to property


The assembly of the bed and installation of accessories and/or optional parts must only be performed by a qualified technician or a person who has received proper training.

- Follow instructions carefully. If you have any question regarding the assembly, contact your Invacare provider.
- The electrical equipment of the bed must not be dismantled or combined with other electrical equipment.
- After each assembly, check that all fittings are properly tightened and that all parts have the correct function.


NOTICE!

- To prevent condensation, the bed should not be used until it has reached operating temperature. See 9 *Technical Data*, page 24.

3.2 Main parts of the bed

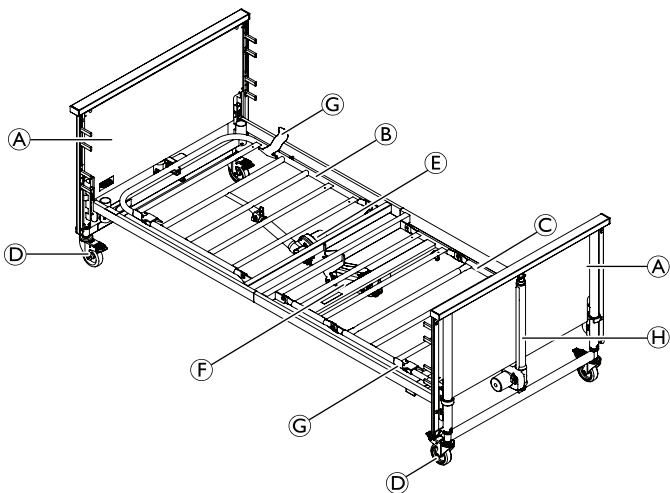

- Ⓐ Bed ends, 2 pcs
- Ⓑ Mattress support, backrest section
- Ⓒ Mattress support, thigh/leg section
- Ⓓ Castor, 4 pcs
- Ⓔ Motor, mattress support backrest section
- Ⓕ Motor and control box, mattress support thigh/leg section
- Ⓖ Mattress retainers, 4 pcs
- Ⓗ Hi/Lo Motor (height adjustment)
Hand control (not shown in image)



Side rails are not delivered in the main configuration. For more information about side rails and other options, contact your Invacare representative.


3.2.1 Other Items

The following items are also provided within the scope of delivery:


Part		Quantity
User manual		1
Side rail gliders		4
Entrapment label		4

3.3 Assembling the bed

1. Untighten thumb screws on both sides.

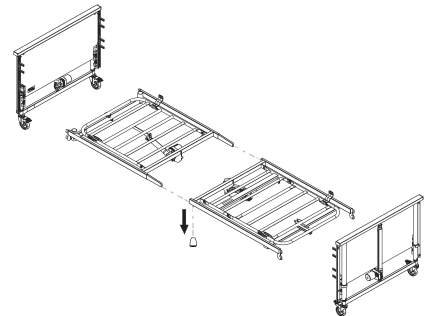


2. Push the inserts on the upper half of the mattress support into the side tubes on the lower half of the mattress support.

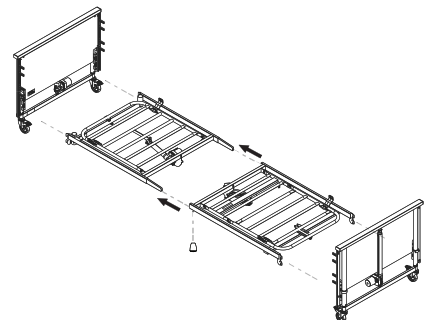

3. Tighten with two thumb screws.
4. Make sure that the four mattress retainers are positioned pointing upwards.

- Turn the locking rings on the bed end to position OPEN.

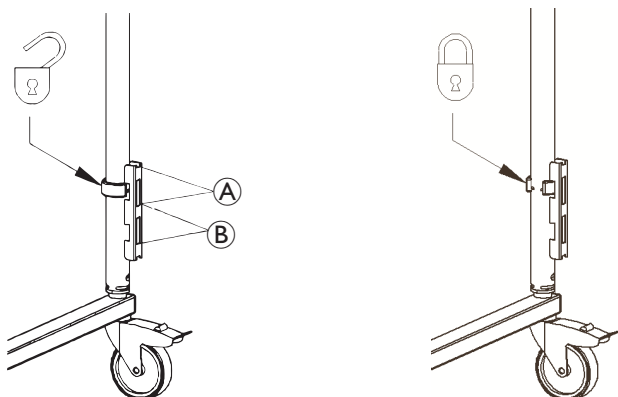

- Latch the brackets of the mattress support either in high position A or low position B to the bed end and press them firmly into position. (It's normal that there's a small space between the upper slot and the bracket)
- Turn the locking rings into Position LOCK.


**CAUTION!**

The locking rings ensure that the mattress support can't be accidentally lifted off the bed ends.

- Make sure that all locking rings are locked and in the upper slot as shown in the picture.
- Make sure that all brackets are attached in the same position (high or low).

3.3.1 Attaching Labels

(Low version only)

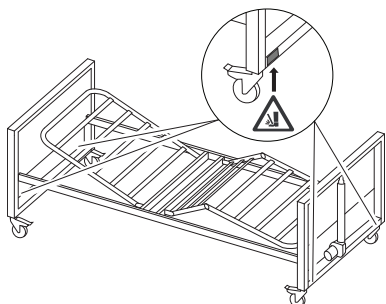


- Place the four foot squeezing warning labels (delivered separately) on the lower sides of the bed ends near the castors.


3.4 Install Rastofix

(optional)


- Press bottom of Rastofix at the foot end of the bed.

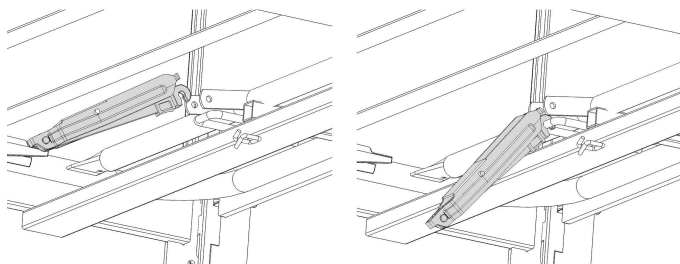

- Squeeze upper end of Rastofix together.

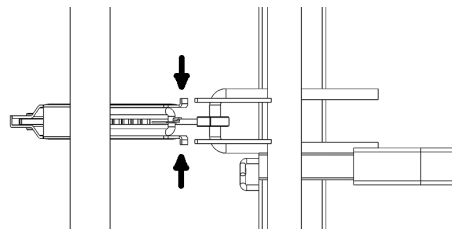

- Rotate Rastofix between tabs and release.

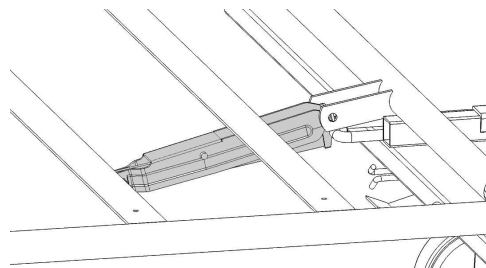


- Slide the plastic cap and lock the bottom of the rastofix.

3.5 Control Box

**NOTICE!**

— The cables must be mounted in such a way they are kept clear off the floor and do not block the castors.

The control box is latched to the thigh motor.

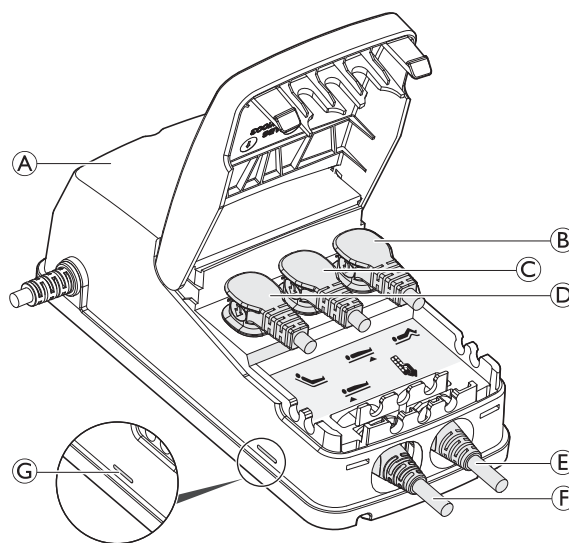

The control box A is provided with a label with symbols showing where to connect the motor plugs:

- Thigh section's motor B.
- Bed end motor, foot end C.
- Backrest motor D
- Hand control E
- Bed end motor, head end F.



The control box is equipped with a green LED G which indicates it is connected to main power. When it is connected to mains power the green LED G is on.

3.6 Wiring




NOTICE!


- The cables must be mounted in such a way that they are kept clear off the floor and do not block the castors.

In order to prevent the cables from being damaged when activating the motors, follow the instructions below.

1. Detach all pre-installed locking clips.
2. Check the labels on the Hi/Lo motor cables (the coiled ones) and plug the cables on the corresponding bed end motor (head or foot).
3. Re-attach and close the locking clips of the motors, and ensure the cables are secure.
4. Manually raise the backrest and connect the loose actuator cable to its motor.

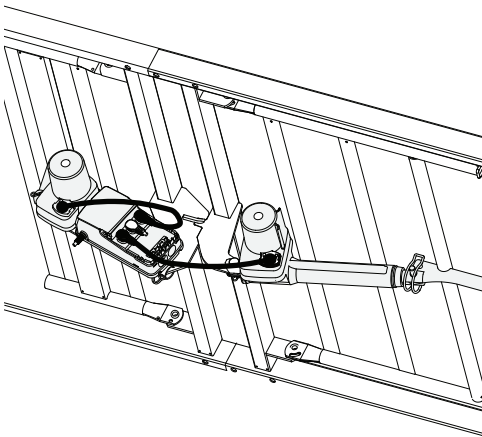


5. Re-attach and close the locking clips of the motor, and ensure the cable is secure.
6. Guide the mains cable through the motor pipe pin and the frame hook on the side of the bed that's closer to the power outlet.

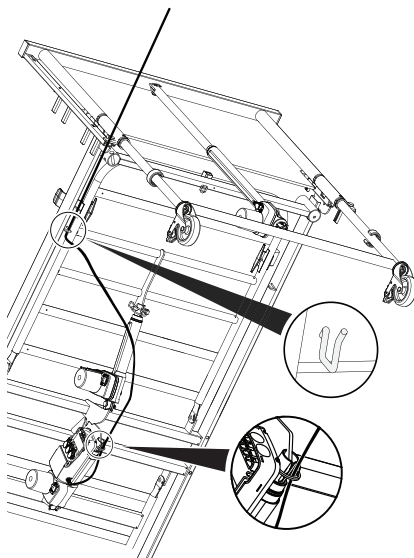


7. Connect the mains cable to the power outlet.
8. Raise the bed to its highest position.

9. Guide the Hi/Lo motor cables through the motor pipe pins.

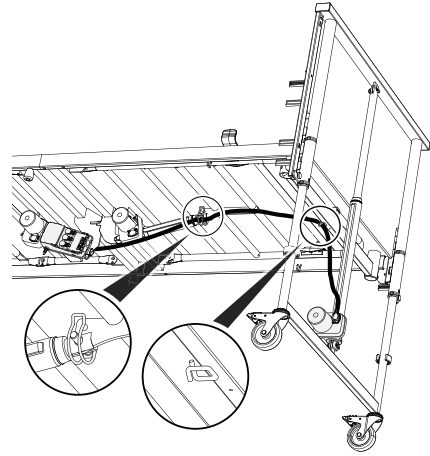

10. Make sure the pipe pins are properly locked.

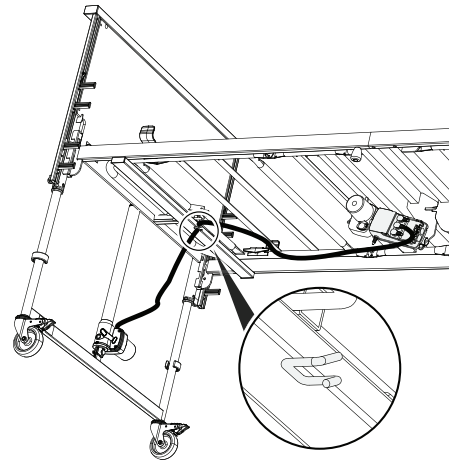

11. Place the end of the coiled part of the Hi/Lo motor cable on the hook at the bed end.
12. Before taking the bed into use, check that all parts run smoothly and that no motor cables are hanging on the floor or being squeezed while moving bed sections.

3.7 Installing side rails



WARNING!

Risk of personal damage

There is a risk of entrapment or squeezing, while assembling or disassembling the side rail.

- Pay attention to your fingers.
- Follow instructions carefully.
- After each assembly, check that all fittings are properly tightened and that the side rail has the correct function.

Removable 3/4 length side rails



WARNING!

Risk of entrapment


There's a risk of entrapment or suffocation between mattress support, side rail and bed end.

- When using removable side rails, always make sure that the distance between the bed end and the side rail's handle upper edge is less than 6 cm in the head end and more than 32 cm in the foot end of the bed.

3.7.1 Scala 2 side rail

Mounting side rail

1.

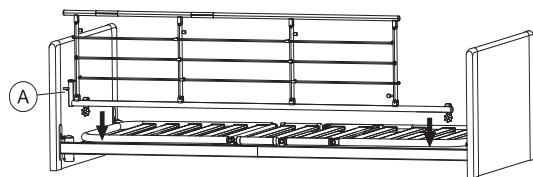


Place the side rail over the top frame with the release system **A** at the head end of the bed.



The fork links on the side rail must be mounted according to the instruction on the side rail.

2.

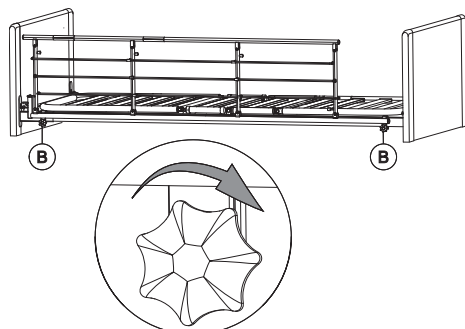

Tighten the finger screws **B** to attach the side rail on the top frame. Make sure it is steady and fastened.

Dismounting side rail

1. Loosen the two finger screws **B** and remove the side rail.

3.7.2 Installing Aria and Bella side rail



WARNING!

Risk of entrapment / suffocation


The user can get trapped or fall out of the bed if the side rail is not correctly mounted or damaged.

- Make sure that all gliding shoes are correctly guided into the bed end guideways.
- Make sure that the locking screws are properly installed at the bottom of the guideways.

The installation of Bella wooden side rail and Aria steel side rail is identical.

Assembling the gliding system

1.

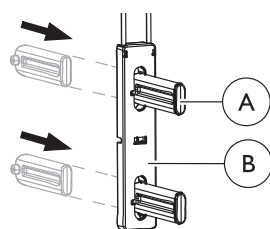


Push the pawl **A** from the backside, through the glider **B** and make sure it locks with an audible click.

2. Do the same for all four pawls in each glider.

Attaching the gliding system and installing the side rail

1. Raise the bed to 1/3 of full height.

2.

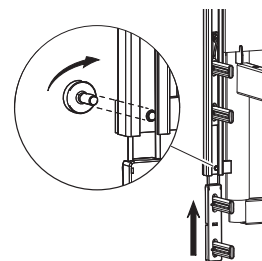

Loosen the screw at the bottom of the guides on one of the bed ends and push the glider halfway up into the guide until it attaches with an audible click.




NOTICE!

Ensure correct positioning of the side rails:


- The side rail with handles must be installed in the upper pawls.
- The side rail handles must be turned towards the outside of the bed.

3.

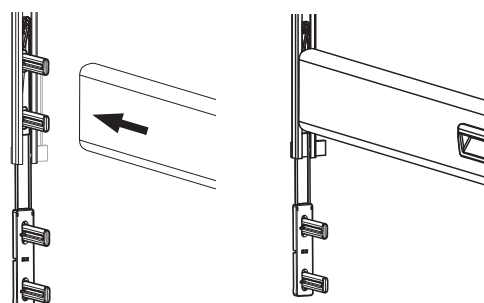


Push the upper side rail onto the two upper pawls.

4.

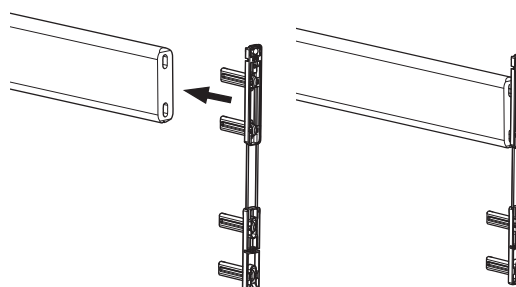

In the other end of the side rail bar; push the two upper pawls into the end of the bar.

5.

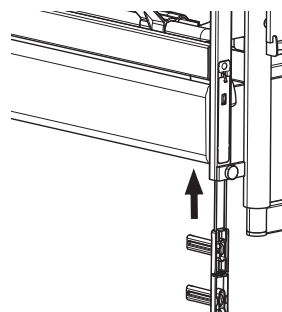


Push the attachment into the guide until it attaches with an audible click (=properly engaged in the lower position).

6. Mount the lower side rail bar in both ends as in step 4.

7. Push the side rail in both ends, all the way up until the upper bar locks in the upper position.

8.

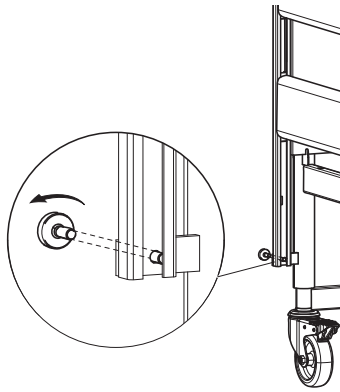

Retighten the screw under the lower side rail bar, in each corner of the bed ends.



Before taking the bed into use; Make sure that the side rail is properly locked with the screw and that the bars run smoothly.

3.8 Installing the mattress support extension

For patients, taller than two meters it is recommended to extend the bed with a mattress support extension. The bed can be extended 15 cm in the foot end of the bed.




WARNING! **Risk of injury**

The user can fall out of bed, get trapped or suffocated.

- Always extend the mattress when extending bed.
- There must be a distance of more than 2,5 cm between the bed end and the mattress bow, to avoid entrapment of fingers.


(optional)

1.

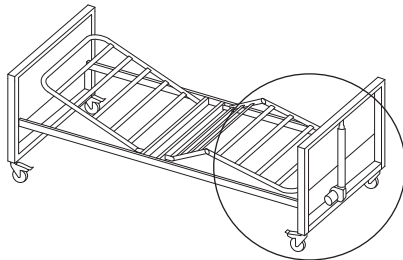


Extension should be installed at the foot end.

2.

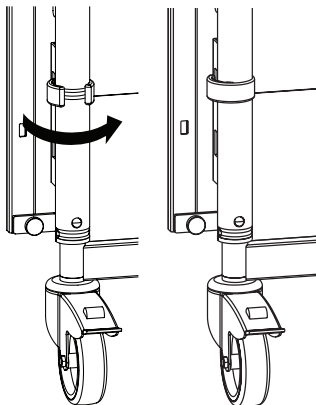


Rotate the clip to unlock.

3.

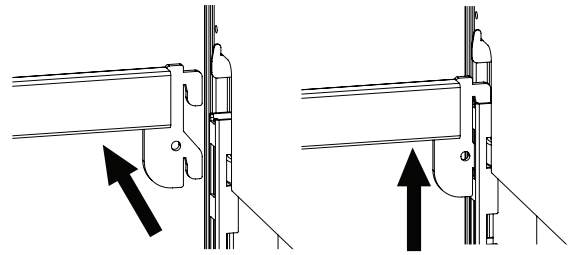



Remove the side rails.

4.

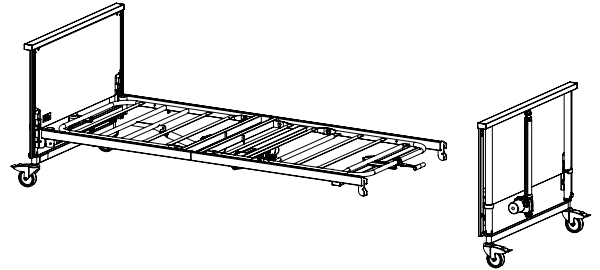


Remove the foot bed end.

5.

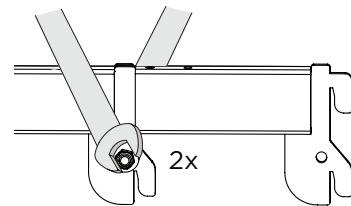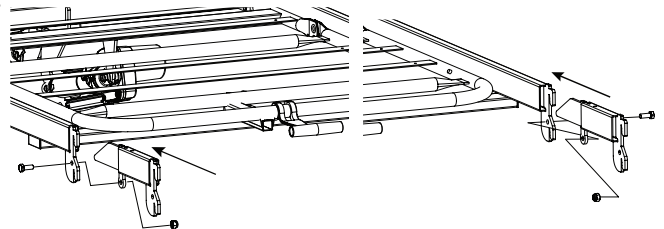



Insert both sides of the mattress support extension into the frame and secure with nuts and bolts.

6.

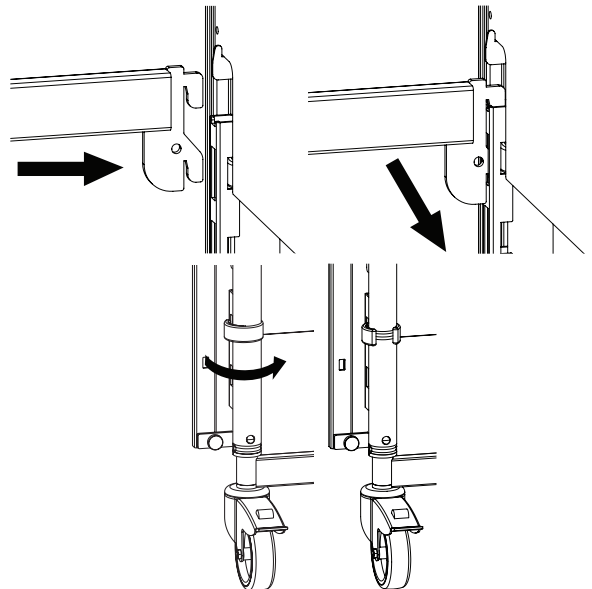


Reinstall the foot bed end and fix it to the brackets.

7. Attach mattress deck extender to the mattress deck.

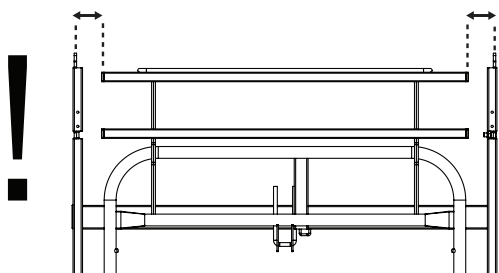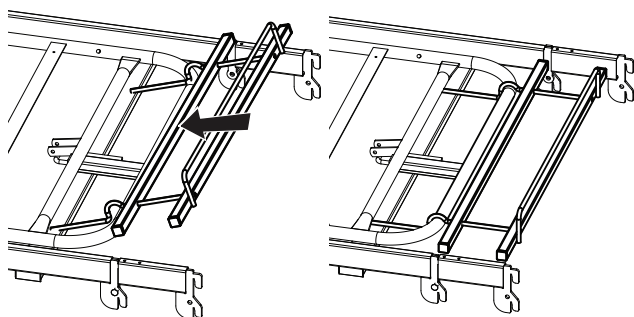

If the bed has a mattress handle assembled on the leg section (see 4.6 *Adjusting the leg section*, page 17) it must be rotated to avoid interfering with the mattress.

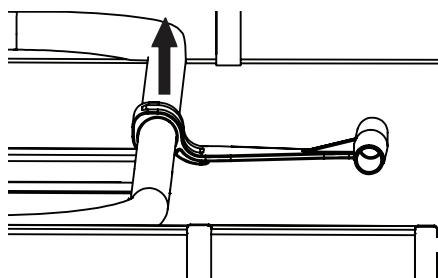


8.




Raise the leg section to facilitate assembly.

9.

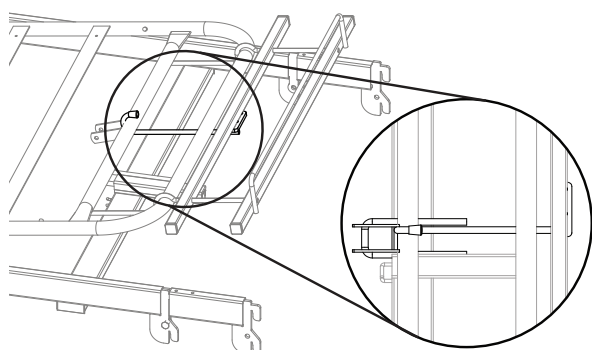


Install the mattress extension hook in the mattress deck, in the middle-right side of the two rastofix tabs.

10.

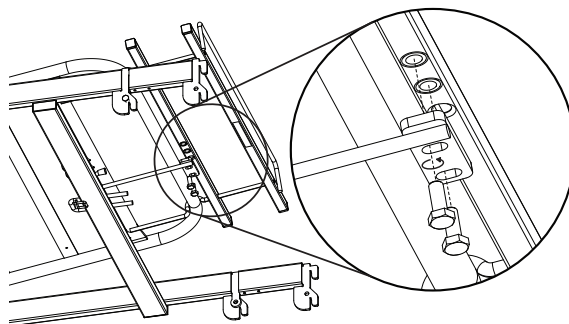

Secure the extension hook to the extension platform, using the available threaded holes.

11.

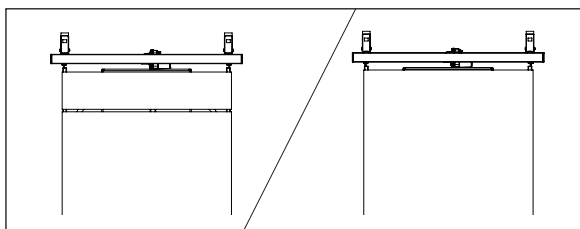

Replace the mattress or put in a fitting mattress piece.



Loose mattress pieces should always be fitted in the foot end of the bed.

12. Mount the side rail(s), if applicable.




NOTICE!

If using the Aria/Bella side rails, only the longer version must be used.

3.9 Dismantle the bed

1. Remove the side rails and the lifting pole.
2. Bring the bed to its lowest position and adjust all mattress support sections to horizontal position.
3. Disconnect the bed from the mains.
4. Disconnect the cables from the bed end motors and remove it from the frame hook and pipe pin routing.

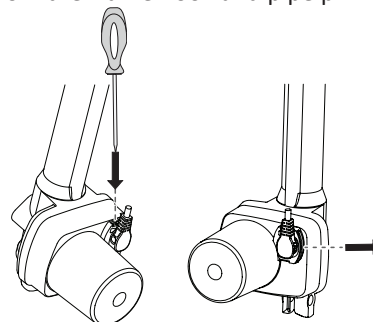

5. Disconnect the cable from the backrest motor.
6. Separate the mattress support from the bed ends.
7. Divide the head and foot end of the mattress support.

4 Operating the bed

4.1 General Safety Information


WARNING!
Risk of personal injury and damage to property.

- The bed must be placed so that the height adjustment is not obstructed by, for example, lifts or furniture.
- Take care that no body parts are being squeezed between fixed parts (such as side rails, bed ends etc) and moving parts.
- The hand control must not be used by children.


WARNING!
Risk of injury

If the bed is used by confused or restless users or users with spasms:

- either, when the bed is equipped with a lockable hand control, lock the hand control functions,
- or make sure that the hand control is out of reach for the user.

4.1.1 Low Heights


CAUTION!
Risk of entrapment or squeezing

Depending on the configuration, the free space between moving and fixed parts of the bed can become very small when the bed is lowered.

- Ensure your feet are not near or under moving parts when lowering the bed.
- Ensure cables are routed properly and can not get squeezed when lowering the bed.


CAUTION!
Risk of injury

Depending on the configuration, the minimum working height of the bed is very low. Using an improper/incorrect posture while working at low working heights can lead to injuries for the carer.

- Be aware of your body posture when attending to the user.

4.1.2 Mattress retainers


WARNING!
Risk of injury

Without properly installed mattress retainers, the mattress can slip sideways and cause the user to fall out of the bed.

- Never use the bed without mattress retainers.
- Always make sure the mattress retainers are properly installed and facing upwards.
- Only use the original mattress retainers.

4.2 Hand control

The hand control can be equipped with either three or four buttons to operate the electrical functions of the bed. The hand control can also be equipped with a locking function.

Backrest section



1. Up: press left side of the button (▲).
2. Down: press right side of the button (▼).

Height adjustment



1. Up: press left side of the button (▲).
2. Down: press right side of the button (▼).

Thigh section



1. Up: press left side of the button (▲).
2. Down: press right side of the button (▼).

Tilt function


CAUTION!
Risk of fatal injury

Tilting with head end down can have fatal effect on users who are sensitive for increased blood pressure in the upper part of the body.

- Only medically trained persons must operate the tilt function.
- Always perform a medical evaluation, before tilting the bed with the user's lower extremities positioned higher than the heart.
- The tilt function is NOT a Trendelenburger function and must not be used for medical treatment.



1. Head end up: press left side of the button (▲).
2. Foot end up: press right side of the button (▼).


If the bed is equipped with anti-tilt function only, tilting with the foot end up is not possible:



1. Head end up: press left side of the button (▲).
2. Horizontal position: bring the bed all the way down (▼). Alternatively, this can be achieved by pressing the height adjustment button to bring it all the way up.

4.2.1 Locking function

The locking function prevents usage of certain function buttons. Regardless of how many buttons your hand control has, they can all be locked individually.

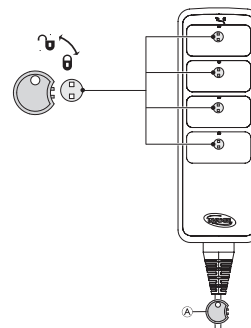

1. Insert key ① in the keyhole located just above the desired function.
2. To lock, turn the key clockwise.
3. To unlock, turn the key anti-clockwise.

4.3 Castors and brakes



CAUTION!

Risk of trapping/squeezing

All brakes are foot-operated.

- Do not release the brake with the fingers.



CAUTION!

Risk of injury


User can fall, while entering or exiting the bed, if brakes are not locked.

- Always lock the brakes before the user is moving in or out of bed or when attending to the user.



Castors might make marks on different types of absorbing floor surfaces - such as untreated or badly treated floors. To prevent marks, *Invacare®* recommends to place a suitable kind of protection between the castors and the floor.

4.3.1 Castor brake

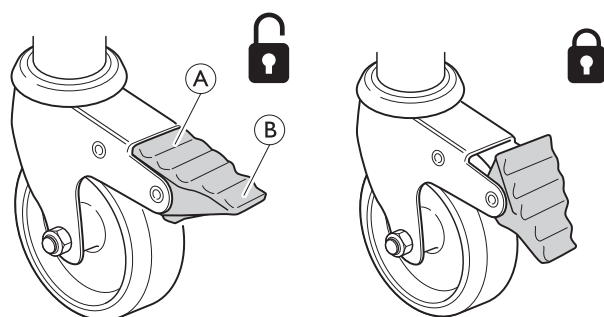

Locking the brake - Step on the outer part **B** of the brake pedal

Unlocking the brake - Step on the inner part **A** of the brake pedal

4.4 Operating side rails



WARNING!

Risk of entrapment or suffocation

There's a risk of entrapment or suffocation between mattress support, side rail and bed end.

- Always ensure correct fitting and tightening of the side rails.



WARNING!

Risk of falling

If it has been decided that the user needs side rails:

- Never leave the user unattended in the bed with the side rail down.

- Make sure the side rail is in its highest position and properly locked while leaving the user unattended.



CAUTION!

Risk of Injury

If the side rail is not locked properly it can fall down.

- Pull/push the top bar of the side rail to ensure that the locking system is properly engaged.



CAUTION!

Risk of Injury

There is a risk of entrapment or squeezing while operating the side rail.

- Pay attention to your fingers and body parts of the patient.
- Never force or drop the side rail while handling it.


4.4.1 Operating Scala 2 side rail




NOTICE!

- When manoeuvring the side rail, place your hands only in the areas marked with the grip label.

1.

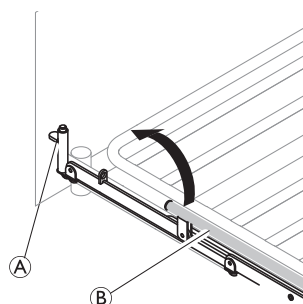

2.

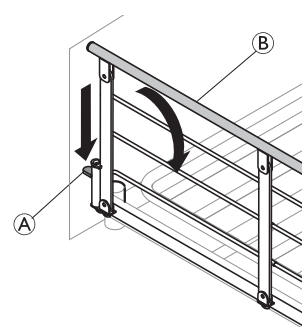

- Up:** Lift and pull the top bar **B** of the side rail towards the end with the locking mechanism (release button) **A**. Ensure, it is locked in place.
- Down:** Press the release button **A** and push the top bar **B** of the side rail away from the locking mechanism.

4.4.2 Aria and Bella side rail


Operating the side rail

Raise the side rail

- Use the two handles on the upper bar to raise the side rail until it locks in the upper position with an audible click.

Lower the side rail

1.

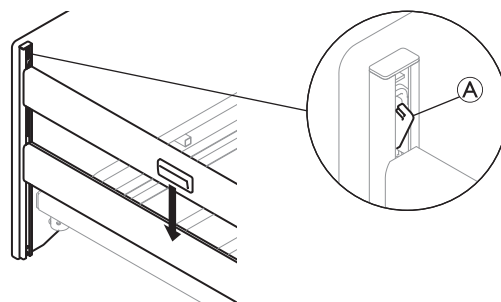

Press the gliding system latch **A** on one end of the bed and lower the side rail to the lowest position.

- Repeat the procedure at the other end of the bed.

4.5 Lifting pole



WARNING!

Risk of injury

The bed can tip if the handle is used, while the lifting pole is turned away from the bed.

- The lifting pole always has to be positioned with the handle hanging over the bed area.
- Do not exceed the maximum load of the lifting pole; 80 kg.

4.5.1 Place the lifting pole

The lifting pole can be placed either on the left or the right side of the head end of the bed.


1. Insert the lifting pole into the lifting pole tube and fix it with the finger screw.



It's not required to tighten the finger screw, in case you want the lifting pole to swing away to the side of the bed.

4.5.2 Adjusting the handle height

The handle height should always be adjusted to the user's need.

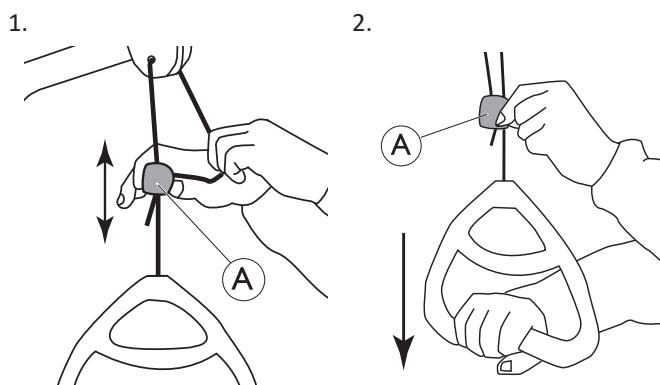

1. Hold the cord out while sliding the plastic cord lock (A) upwards or downwards until the handle reaches the preferred height.
2. Lock the handle by pressing back the cord in the lock (A) and pull the handle downwards.




NOTICE!

After adjusting the handle height:

- Check that the two cords above the cord lock are parallel and inside the cord lock.
- Make sure the cord is properly locked by pulling the handle hard.

4.6 Adjusting the leg section

Operate the leg section by lifting the mattress handle:

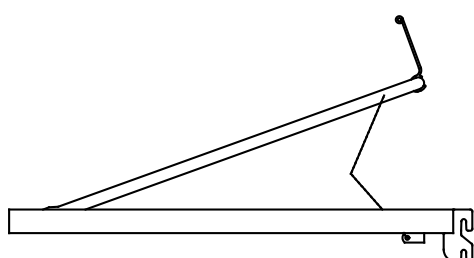

1. **Up:** Lift the mattress handle on the leg section.
2. **Down:** Lift the mattress handle on the leg section right up and then lower the leg section.



If the bed is extended in length the mattress handle will not be reachable, and the leg section must be operated by directly lifting the leg section laterally.

4.7 Emergency positions

In case of a medical emergency, it might be required to bring all or certain mattress support sections into a flattened position - e.g. a flattened back section for cardiopulmonary resuscitation (CPR).

To bring a mattress support section into a flattened position,

- use the corresponding function on the hand control
- OR
- in case of power failure or urgent need for action, use the manual emergency release and lower the mattress support section.

Refer to 4.7.1 *Emergency release of a mattress support section*, page 17.

4.7.1 Emergency release of a mattress support section

In case of a power- or motor failure, an emergency release of the back, thigh or leg section could be necessary. An emergency release of the height adjustment is NOT possible.



CAUTION!

Risk of injury


- A minimum of two persons is required for an emergency release of a mattress support section.
- When releasing a mattress support section, it might lower fast. Do not reach under the mattress support while lowering it.



NOTICE!

- Before an emergency release of the mattress support, remove the plug from the mains socket.

1. Both persons hold the mattress section.
2. One of them locates the motor in question and pulls out the safety pin.

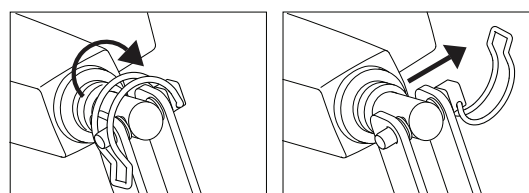

3. Both persons slowly lower the mattress section until it is completely down.

4.7.2 Transportation and Storage

The bed can be transported or stored in assembled or disassembled state.

See 3.9 *Dismantle the bed, page 14* for detailed instructions on how to disassemble the bed.

To transport or store the bed in its assembled state:

1. Disconnect the power cable from the mains socket.
2. Coil up the power cable and ensure that it is clear of the floor during transport, or storage, so it is not damaged.

5 Options and accessories

5.1 List of available options



CAUTION!

Compatibility of bed with optional parts

Improper combination of this product with non-Invacare optional parts may cause injury or damage.

- A risk assessment must be carried out by a professional, prior to combining an Invacare bed with other manufacturers' products.
- In case of doubt regarding the use of an Invacare bed with products from other manufacturers, contact your Invacare provider for information.



NOTICE!

- Use only original options and spare parts. Options and spare parts lists and extra user manuals can be ordered from Invacare.



Due to regional differences, refer to your local Invacare website or catalog for details about available options or contact your Invacare provider.


- Side rails
 - Scala Basic 2
3/4 length collapsible steel side rail (168 cm x 40 cm), mounted on the side tube. Release button in the head section
 - Scala Basic Plus 2
3/4 length collapsible steel side rail (168 cm x 40 cm), mounted on the side tube. Release button in the head section
 - Scala Medium 2
3/4 length collapsible steel side rail (165 cm x 46 cm), mounted on the side tube. Release button in the head section
 - Scala Decubi 2
3/4 length collapsible steel side rail (168 cm x 54 cm), mounted on the side tube. Release button in the head section
 - Bella
Full length collapsible wood side rail (205 cm x 40 cm), mounted on the bed end guideways. Release buttons in both ends
 - Bella extended
Full length collapsible wood side rail (220 cm x 40 cm), mounted on the bed end guideways. Release buttons in both ends
 - Aria
Full length collapsible steel side rail (206 cm x 40 cm), mounted on the bed end guideways. Release buttons in both ends
 - Verso HC
3/4 length collapsible steel side rail (150 cm x 40 cm), mounted on the side tube. Release button in the leg section.
- Mattress support extension (foot end) – 15 cm
- Lifting pole
- Fixed support handles – 25 cm x 30 cm, 25 cm x 80 cm, 40 cm x 30 cm, 40 cm x 50 cm or 40 cm x 80 cm

- Hand controls
 - with back, thigh and height adjustment
 - with back, thigh and height adjustment, and tilt and anti-tilt functions
 - with back, thigh and height adjustment, and anti-tilt function
- Soft Tilt – Patient repositioning system mounted on the mattress platform.
- Transport fitting


5.2 Transport brackets

Use the transport brackets to store and transport the bed:


1.

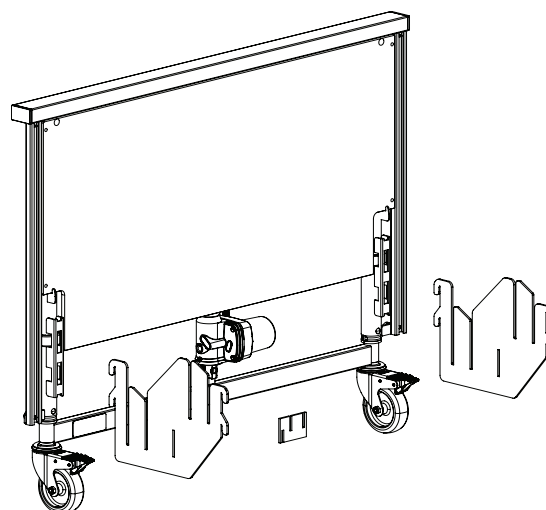


2.

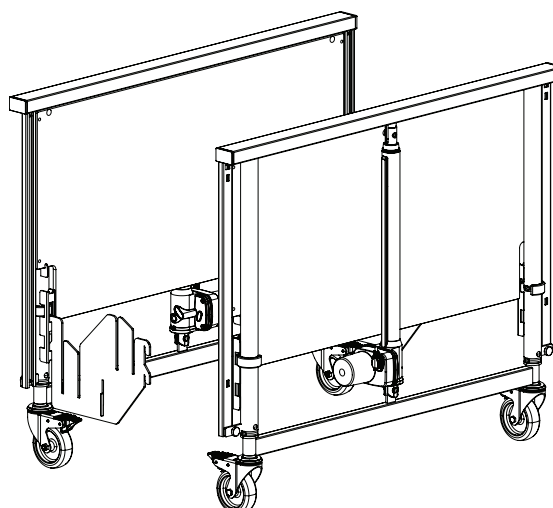


3.

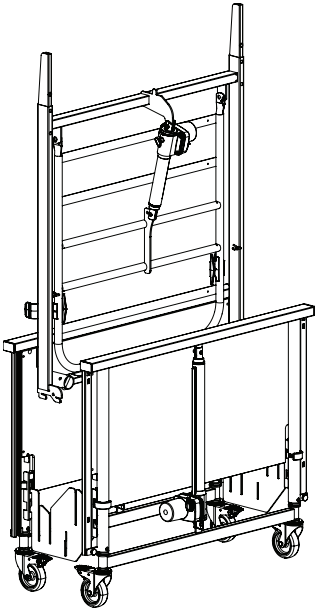


4.

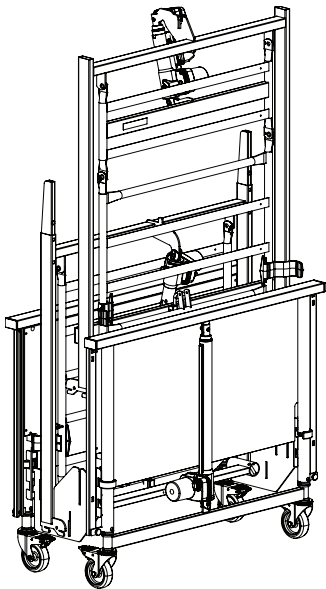

5.

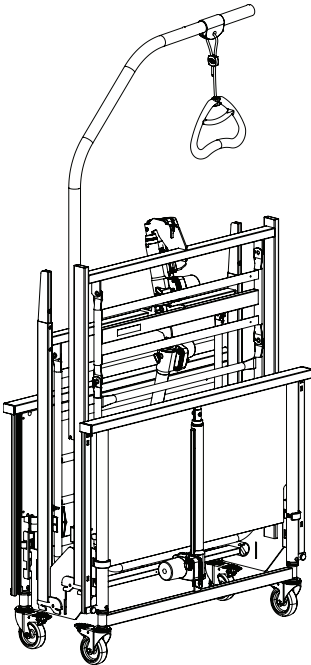

6.

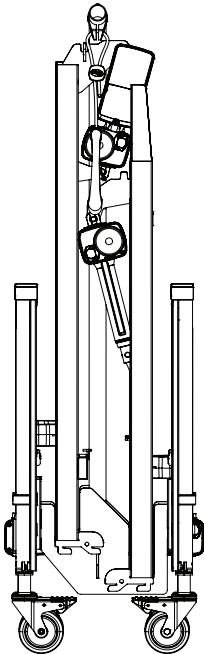

6 Maintenance

6.1 General maintenance information


CAUTION!
Risk of injury or damage

- Do not perform any maintenance or service procedures while the product is in use.

Follow the maintenance procedures described in this manual to keep your product in continuous operation.

6.2 Daily inspections


WARNING!
Risk of injury or damage

Damaged or worn parts can affect the safety of the product.

- The product should be checked each time it is used.
- Do not use the product if damage is found or you question the safety of any part of the product. Contact your Invacare provider immediately and ensure the product is not used until repairs are made.

6.2.1 Daily Inspection Checklist

- ☐ Visually inspect the bed. Check all parts for external damage or wear.
- ☐ Check all hardware, attachment points, stressed parts and any pivot points for signs of wear, cracking, fraying, deformation or deterioration.
- ☐ Check the hand control and all actuators for smooth operation.

6.3 General servicing information


NOTICE!

- The mattress support must be supported during service inspections, to prevent accidental lowering.
- Only personnel who have received the necessary instructions or training may perform service and maintenance of the bed.
- After reconditioning the bed, or if bed functions change, service must be carried out according to the check list.

A service contract can be made in the countries, where Invacare has its own sales company. In certain countries Invacare offers courses in service and maintenance of the bed. Spare parts lists and additional user manuals are available from Invacare.

Before Use

- Ensure that all manual and electrical parts functions correctly and are in a secure state.

After three months

- Ensure that all manual and electrical parts are functioning, and tighten bolts, screws, nuts, etc.

Every year

- We recommend a safety test comprising the actuators' performance and mechanical state.

Every second year

- We strongly recommend a service according to the below checklist after two years of normal use and then every second year.



Actuators, hand control and control unit are serviced by exchanging the faulty part.

6.3.1 Checklist maintenance

Checkpoints

- ☐ Visual inspection of all parts of the bed (no deformations)
- ☐ Circlips, cotter pins and plastic fixing ring - properly locked and intact.
- ☐ Screws - tightened.
- ☐ Weldings - intact (no wear and tear)
- ☐ Castors (rolling smoothly)
- ☐ Castor fittings - tightened.
- ☐ Castor brakes - locking properly.
- ☐ All motors - running properly (with regular speed and at low noise).
- ☐ Wires - correctly wired and undamaged.
- ☐ Electric plugs - undamaged.
- ☐ Side rail locking and moving system - properly locking and running smoothly.
- ☐ Accessories and/or optional parts - correctly assembled and correct function.
- ☐ Damaged coating - repaired.
- ☐ Leakage of grease - check for grease.


CAUTION!
Risk of injury or damage to property

- Grease leaks can lead to accidental falls or fire.
- Contact service provider if leaks of grease or other residue is detected.

6.4 Inspection after relocation - Prepare for new user

- ! NOTICE!**
When the bed has been relocated; before given to a new user it has to be thoroughly inspected.
- Inspection must be done by a trained professional.
 - For regular maintenance see maintenance chart.

6.5 Cleaning and Disinfection

6.5.1 General Safety Information

- ! CAUTION!**
Risk of Contamination
- Take precautions for yourself and use appropriate protective equipment.

- ! CAUTION!**
Risk of Electric Shock and Product Damage
- Switch off the device and disconnect from mains, if applicable.
 - When cleaning electronic components consider their protection class regarding water ingress.
 - Make sure that no water splashes to the plug or the wall outlet.
 - Do not touch the power socket with wet hands.

- ! NOTICE!**
Wrong fluids or methods can harm or damage the product.
- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
 - Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
 - Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
 - Always make sure that the product is completely dried before taking into use again.

- i** For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

6.5.2 Cleaning Intervals

- ! NOTICE!**
Regular cleaning and disinfection enhance smooth operation, increases the service life and prevents contamination.
- Clean and disinfect the product:
- regularly while in use,
 - before and after any service procedure,
 - when it has been in contact with any body fluids,
 - before using it for a new user.

6.5.3 Cleaning by Hand

All components (excluding removable textiles)

- Cleaning agent: We recommend using a mild detergent with neutral or near neutral pH (5-9).
- i** Commercially available consumer products like hand dishwashing liquid or all-purpose cleaners can be used. Read the instructions on the label of your cleaning agent and use it at the indicated concentration.
- Max. water temperature: 40 °C
1. Wipe surfaces thoroughly with a soft cloth slightly moistened with detergent solution until all visible soil is removed.
 2. Remove all detergent residue with a clean damp cloth.
 3. Wipe dry using a clean soft cloth.

Removable textiles (including upholstery and mattresses)

- See attached label on each product.

6.5.4 Disinfection Instructions

In Domestic Care

- Disinfectant: We recommend using an alcohol-based surface disinfectant (with 70-90% alcohol).
- i** Read the instructions on the label of your disinfectant. It provides information on the activity spectrum (bacteria, fungi and/or viruses), material compatibility and the correct exposure time.
1. Ensure the surfaces are cleaned before disinfection.
 2. Moisten a soft cloth and wipe-disinfect all accessible surfaces and keep them moistened for the exposure time stated on the label of the disinfectant.
 3. Allow the product to air-dry.

In Institutional Care

Follow your in-house disinfection procedures and only use the disinfectants and methods specified therein.

6.6 Lubrication

We recommend lubricating the bed according to the following instruction after service and cleaning or whenever it is necessary to reduce friction.

Part of bed	Lubrication method
Points of rotation in mattress support	Oil (medically clean)
Motor attachment points on mattress support	Oil (medically clean)
Motor bearings and tension rods	Oil (medically clean)

- i** The wooden side rails gliding system must not be lubricated with oil - otherwise the wooden bars will move sluggishly.

For ordering of correct oil, contact your Invacare provider.

7 After Use

7.1 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

7.2 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *6 Maintenance, page 21*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

8 Troubleshooting

8.1 Troubleshooting electrical system

Symptom	Possible cause	Remedy
Mains indicator does not light up	Mains are not connected	Connect mains
	Fuse in the control unit is blown	* Replace the control unit
	Control unit is defective	* Replace the control unit
Mains indicator lights up, but the motor is not running. The relay in the control unit makes a clicking noise.	Motor plug is not fully inserted into the control unit.	Insert the motor plug properly into the control unit
	The motor is defective.	* Replace the motor
	Motor cable is damaged.	* Replace the cable
	Control unit is defective	* Replace the control unit
Mains indicator lights up, but the motor is not running. No relay sound is heard from the control unit.	Control unit is defective	* Replace the control unit
	Hand control is defective	* Replace the hand control
Control unit is in order except for one direction on one channel.	Control unit is defective	* Replace the control unit
	Hand control is defective	* Replace the hand control
Motor is running, but the piston rod does not move.	Motor is damaged	* Replace the motor
The motor cannot lift full load.		
Motor noise, but no movement of piston rod.		
Piston rod operates inwards and not outwards.		
* Service and maintenance of the bed must only be performed by personnel who have received the necessary instruction or training.		
WARNING! Risk of personal injury and damage to the product. — The bed must be unplugged from the main power source before opening or repairing electrical parts.		


9 Technical Data

9.1 Materials

Mattress Platform	Steel (powder-coated)
Bed Ends	Steel (powder-coated) and MDF
Side Rails	Steel (powder-coated) or Wood
Lifting Pole	Steel (powder-coated), PP and POM
Actuators housing, Hand control, Control box, Castors and other Plastic parts	Material according to marking (PA, PP, PE, ABS)
Bolts and nuts	Steel (Zinc-plated)

9.2 Dimensions bed

All measurements are stated in cm. All angles are stated in degrees. All measurements and angles are stated without tolerances.

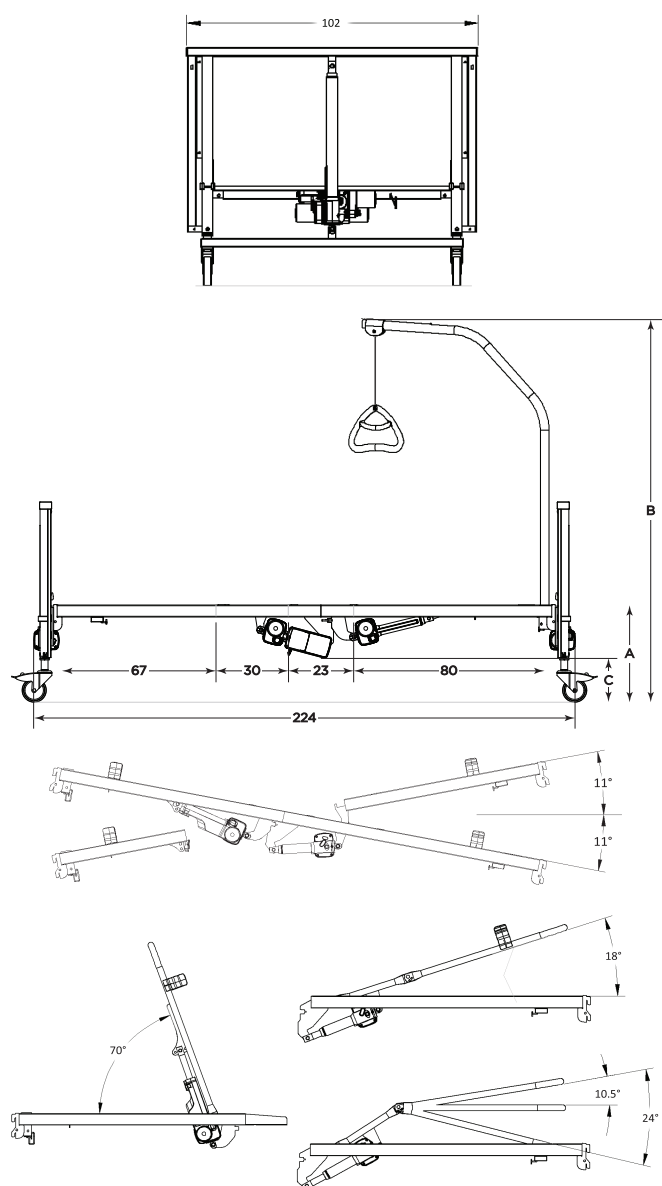

Height	Medley Ergo	
	Low	High
A	33 – 73 cm	40 – 80 cm
B	160 – 201 cm	166 – 207 cm
C	12 – 55 cm	19 – 62 cm

Height	Medley Ergo Low	
	Low	High
A	21 – 61 cm	28 – 68 cm
B	146 – 187 cm	153 – 194 cm
C	0,5 – 42 cm	7,5 – 49 cm

9.3 Weights

	Medley Ergo	Medley Ergo Low
 Max. patient weight (provided that the weight of the mattress and the accessories and/or optional parts do not exceed 35 kg)	145 kg	
 Max. safe working load (patient + accessories and/or optional parts)	180 kg	
Total weight*	64,0 kg	
Upper part of mattress support with wood slats (= heaviest part)	17,0 kg	
Upper part of mattress support, with steel slats	16,0 kg	
Lower part of mattress support with wood slats	16,0 kg	
Lower part of mattress support, with steel slats	15,0 kg	
Bed end (1 piece)	14,7 kg	13,7 kg
Lifting pole	4,2 kg	
Transport brackets	2,0 kg	
Mattress support extension (15 cm)	1,9 kg	

*Complete bed, incl. mattress support (upper and lower), steel slats, main cord, mattress retainers and hand control.

9.4 Mattress dimensions

Allowed mattress sizes, depending on the side rail

Side rail	Mattress size		
	Height [cm]	Width [cm]	Length [cm]
Bella /Aria high ¹⁾	6 – 12	85,5 – 90	200 – 208
Bella /Aria low ²⁾	12 – 18	85,5 – 90	200 – 208
Scala Basic 2	10 – 18	85,5 – 90	200 – 208
Scala Basic Plus	10 – 18	85,5 – 90	200 – 208
Scala Medium	10 – 22	85,5 – 90	200 – 208
Scala Decubi 2	23 – 31	85,5 – 90	200 – 208
Verso HC	10 – 18	85,5 – 90	200 – 208

1) Mattress support mounted in high position.

2) Mattress support mounted in low position.

For beds equipped with a mattress support extension, add 15 cm.

Min. density: 38 kg/m

9.5 Electrical data

Voltage supply	U _{in} = 230 Volt, AC, 50/60 Hz (AC = Alternating current)
Maximum current input	I _{in max.} = 1.5 A
Intermittent (periodic motor operation)	Int = 10%, max. 2 min / 18 min
Insulation class	CLASS II equipment
Type B Applied Part	Applied Part complying with the specified requirements for protection against electrical shock according to IEC60601-1.
Sound level	45 to 50 dB (A)
Degree of protection	The electrical system is IPX4 – protected against water splashed against the components from any direction.  Height adjustment motors are IPX4. The mattress platform motors, control unit and hand control are IPX6. IPX6 – These electrical components are protected against water projected in powerful jets from any direction.

 The bed has no isolator (main switch). If the bed needs to be electrically disconnected, it has to be unplugged from the mains socket.

9.6 Environmental conditions

	Storage and Transportation	Operation
Temperature	-10°C to +50°C	+5°C to +40°C
Relative humidity	20% to 80% - not condensing	
Atmospheric pressure	700 hPa to 1060 hPa	

 Allow the product to reach operation temperature before usage:

- Warming up from minimum storage temperature might take up to 24 hours.
- Cooling down from maximum storage temperature might take up to 24 hours.

10 Electromagnetic Compatibility

10.1 General EMC information

Medical Electrical Equipment needs to be installed and used according to the EMC information in this manual. This product has been tested and found to comply with EMC limits specified by IEC/EN 60601-1-2 for Class B equipment. Portable and mobile RF communications equipment can affect the operation of this product.

Other devices may experience interference from even the low levels of electromagnetic emissions permitted by the above standard. To determine if the emission from this product is causing the interference, run and stop running this product. If the interference with the other device operation stops, then this product is causing the interference. In such rare cases, interference may be reduced or corrected by the following:

- Reposition, relocate, or increase the separation between the devices.

10.2 Electromagnetic emission

Guidance and manufacturer's declaration

This product is intended for use in the electromagnetic environment specified below. The customer or the user of this product should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group I	This product uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	This product is suitable for use in all establishments including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations /flicker emissions IEC 61000-3-3	Complies	

10.3 Electromagnetic Immunity

Guidance and manufacturer's declaration

This product is intended for use in the electromagnetic environment specified below. The customer or the user of this product should assure that it is used in such an environment.

Immunity test	Test / Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrostatic transient / burst IEC 61000-4-4	± 2 kV for power supply lines; 100 kHz repetition frequency ± 1 kV for input / output lines; 100 kHz repetition frequency	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line to line ± 2 kV line to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	< 0% U_T for 0,5 cycle at 45° steps 0% U_T for 1 cycles 70% U_T for 25 / 30 cycles < 5% U_T for 250 / 300 cycles	Mains power quality should be that of a typical commercial or hospital environment. If the user of this product requires continued operation during power mains interruptions, it is recommended that the product is powered from an un-interruptible power supply or a battery. U_T is the a. c. mains voltage prior to application of the test level.

Immunity test	Test / Compliance level	Electromagnetic environment – guidance
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Conducted RF IEC 61000-4-6	3 V150 kHz to 80 Mhz	Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which this product is used exceeds the applicable RF compliance level above, this product should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating this product.
Radiated RF IEC 61000-4-3	6 Vin ISM & amateur radio bands 10 V/m80 Mhz to 2,7 GHz 385 MHz - 5785 MHz test specifications for immunity to RF wireless communication equipment refer to table 9 of IEC 60601-1-2	Interference may occur in the vicinity of equipment marked with the following symbol:  Portable and mobile RF communications equipment should be used no closer than 30 cm to any part of this product including cables.



These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

10.3.1 EMC test specifications

IEC 60601-1-2 — Table 9

Test Frequency (MHz)	Band ^{a)} (MHz)	Service ^{a)}	Modulation ^{b)}	Immunity test level (V/m)
385	380 - 390	TETRA 400	Pulse modulation ^{b)} 18 Hz	27
450	430 - 470	GMRS 460, FRS 460	FM ^{c)} ± 5 kHz deviation 1 kHz sine	28
710 745 788	704 - 787	LTE Band 13, 17	Pulse modulation ^{b)} 217 Hz	9
810 870 930	800 - 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation ^{b)} 18 Hz	28
1720 1845 1970	1700 - 1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation ^{b)} 217 Hz	28
2450	2400 - 2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation ^{b)} 217 Hz	28
5240 5500 5785	5100 - 5800	WLAN 802.11 a/n	Pulse modulation ^{b)} 217 Hz	9



If necessary to achieve the immunity test level, the distance between the transmitting antenna and the Medical Electrical Equipment or System may be reduced to 1 m. The 1 m test distance is permitted by IEC 61000-4-3.

^{a)}For some services, only the uplink frequencies are included.

^{b)}The carrier shall be modulated using a 50 % duty cycle square wave signal.

^{c)}As an alternative to FM modulation, the carrier may be pulse modulated using a 50 % duty cycle square wave signal at 18 Hz. While it does not represent actual modulation, it would be worst case.



United Kingdom & Ireland:

Invacare Limited
Pencoed Technology Park, Pencoed
Bridgend CF35 5AQ
Tel: (44) (0) 1656 776 200
uk@invacare.com
www.invacare.co.uk



Invacare UK Operations Limited
Unit 4, Pencoed Technology Park,
Pencoed
Bridgend CF35 5AQ
UK



Invacare Portugal Unipessoal, Lda
Rua Estrada Velha 949
4465-784 Leça do Balio
Portugal

60128481-F 2025-08-07



Making Life's Experiences Possible®



Yes, you can.®