Invacare® Cascata

H720T4C

en	Mobile Toilet Chair User Manual	3
de	Mobiler Toilettenstuhl Gebrauchsanweisung	5
fr	Chaise toilette mobile Manuel d'utilisation	9
it	Sedia da toilette mobile Manuale d'uso	12
es	Silla de inodoro móvil Manual del usuario	15
pt	Cadeira sanitária móvel Manual de utilização	18
nl	Mobiele toiletstoel Gebruiksaanwijzing	21

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for future reference.

Yes, you can.®

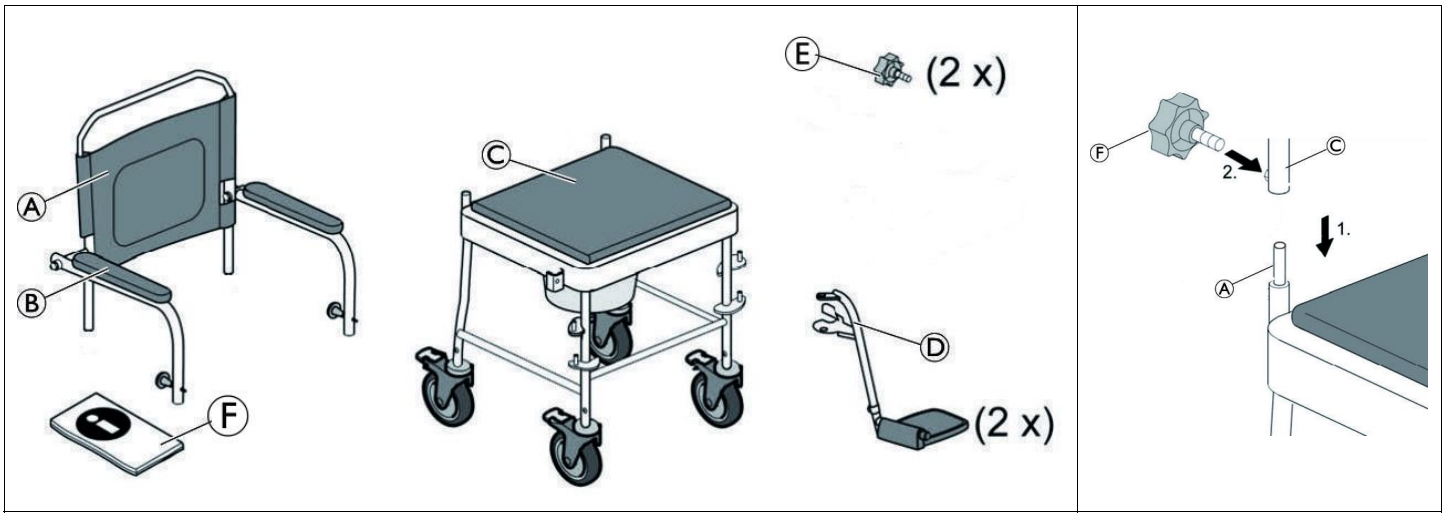
Fig. 1

Fig. 2

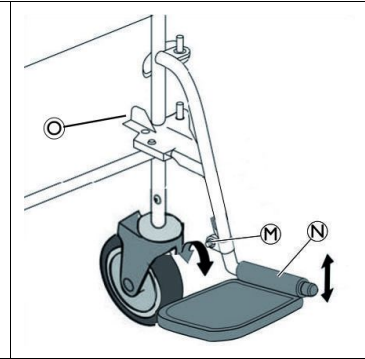
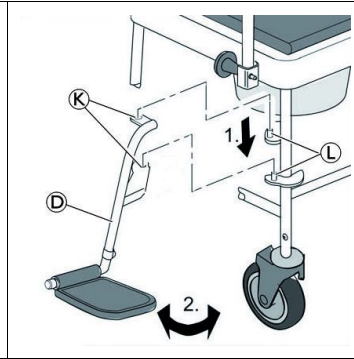
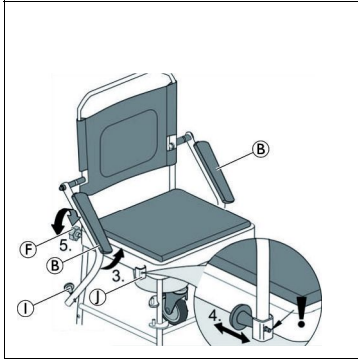
Fig. 3

Fig. 4

Fig. 5

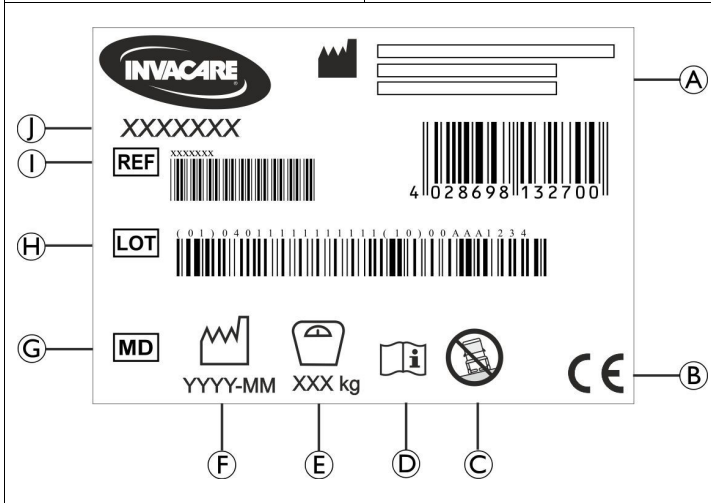
Fig. 6

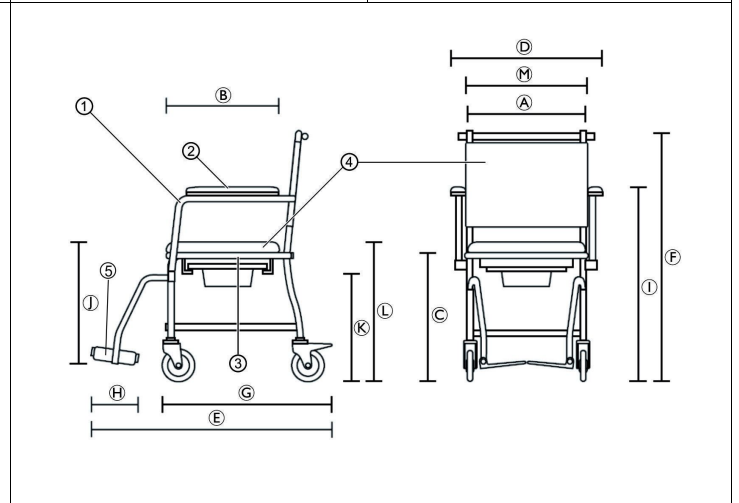
Fig. 7

1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

3.1 Product-specific Standards

The product has been tested and conforms to EN 12182 (Assistive products for persons with disability) and related standards. For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

4 Service Life

The expected service life of this product is three years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

6 Safety

6.1 Safety Information

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

WARNING!

Risk of Falling!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Do not use in shower.
- Keep product away from open flames.
- Do not place smoldering cigarettes on the product.
- Do not stand on this product.
- Do not use as a climbing aid.
- Do not lift the product when under load.
- Only use the toilet chair on a level ground and indoors.

WARNING!

Risk of Tipping!

- Never stand on the footrests.
- Do not lean excessively forwards.

CAUTION!

Risk of Burning Skin!

The product components can heat up when exposed to external sources of heat.

- Do not expose the product to direct sunlight before use.

WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

CAUTION!

Risk of Injury

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalog or website for available options or contact your Invacare distributor. See addresses at the end of this document.

6.2 Labels and Symbols on the Product (Fig. 6)

(A)	Manufacturer	(F)	Production date
(B)	European Conformity	(G)	Medical Device

Ⓒ	For use on level ground only	Ⓗ	Lot number
Ⓓ	Read user manual	Ⓛ	Reference number
Ⓔ	Maximum user weight	Ⓜ	Product designation

 The identification label is attached to the left rear tube of the product.

7 Product Description

Invacare Cascata H720T4C is a mobile toilet chair composed of flip down plastic padded armrests, a padded backrest, with swing-away footrest, a padded seat plate with toilet cut-out and with four braked castors. A toilet pan with lid can be placed under the seat.

8 Scope of Delivery (Fig. 1)

- Ⓐ Backrest
- Ⓑ Armrest (2x) (preassembled)
- Ⓒ Seat frame with swivel castors and toilet pan holder with toilet pan
- Ⓓ Footrest (2x)
- Ⓔ Star head screw (2x)
- Ⓕ User manual

9 Intended Use

This product is intended to be used as a toilet aid providing stable seating position while going to the toilet.

Users are any adults and adolescents with restricted or limited mobility (e.g.: legs and/or hips). If used without an assistant, the users must be able to keep their own balance.

Indication

Restricted or limited mobility (e.g.: legs and/or hips) or having balance issues to stand up or walk due to transitional or permanent disabilities.

There are no contraindications known when using the product as intended.

The maximum user weight: 130 kg

10 Assembling the Product (Fig. 2 — 5)

CAUTION!
Risk of Trapping
– Take care during assembly.

1. Guide the backrest tube Ⓒ on both sides, from the top onto the seat frame. (Fig 2) Ⓐ.
2. Tighten the star head screw Ⓔ (hand-tight). (Fig. 3)
3. Swivel armrests Ⓑ on both sides upwards until the spring pin Ⓛ latches into the hole of the lock Ⓜ on the seat frame. (Fig 3)

CAUTION!
Risk of Falling
– Make sure that the spring pin latches on both sides with an audible click.

4. Place the footrests Ⓓ on both sides through the lateral holes Ⓚ into the attachment pins Ⓛ. (Fig 4)
5. Swivel both footrests to the front.

Adjust the height of the footrests (Fig. 5)

1. Pull up the lever Ⓞ and release the footrest by lifting it upward.
2. Loosen screw with nut Ⓜ with an Allen key and move the lower part of the footrests Ⓨ up or down. Tighten the nut again.
3. Place the footrest back in by proceeding in reverse order.

The device is dismantled by following the assembly procedure in reverse order.

11 Usage

Releasing the Armrests

1. Pull the spring pin knob Ⓛ. The armrest is released from its fixation Ⓛ and swivels downward. (Fig. 3)

Transfer to and from the Chair

CAUTION!
Risk of Trapping/Squeezing
All castor locks are foot-operated.
– Do not operate the castor locks by hand.

1. Lock the castor lock by stepping on the rear end of the pedal until it engages.
2. Flip up the footrests.
3. Alternative to 2.: Press the black lever to swivel the footrests to the side. (Fig. 5)

WARNING!
Risk of Slipping and Rolling
– Lock all castors with levers using the foot levers before sitting down on the toilet chair or standing up from it, as well as in the parking position.

WARNING!
Risk of Tipping!
– Swivel the footrests to the side or fold up the foot plates before sitting down or standing up.

Inserting/Removing the Toilet Pan

The toilet pan can be inserted/removed with or without cover.

1. Insert the upper rim of the toilet pan from the rear side between the seat and the brackets under the seat.

To remove the toilet pan, repeat step 1 in reverse order.

The toilet pan can be lifted and carried by its handle.

12 Maintenance

12.1 General Maintenance Information

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

12.2 Cleaning and Disinfection

General Safety Information

CAUTION!
Risk of Contamination
– Take precautions for yourself and use appropriate protective equipment.

! IMPORTANT!

- Wrong fluids or methods can harm or damage the product.
- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
 - Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
 - Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
 - Always make sure that the product is completely dried before taking it into use again.

Cleaning Intervals**! IMPORTANT!**

- Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.
Clean and disinfect the product
- regularly while it is in use,
 - before and after any service procedure,
 - when it has been in contact with any body fluids,
 - before using it for a new user.

Cleaning Instructions**! IMPORTANT!**

- Clean by hand
- Maximum cleaning temperature 60 °C

1. Clean the product by wiping down all generally accessible surfaces.
2. Dry the parts with a cloth.

Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

13 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *12 Maintenance, page 4*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

14 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

15 Technical Data (Fig. 7)

Dimensions			
Ⓐ	445 mm	Ⓛ	750 mm
Ⓑ	455 mm	Ⓜ	400 – 510 mm
Ⓒ	490 mm	Ⓝ	425 mm
Ⓓ	580 mm	Ⓟ	510 mm
Ⓔ	930 mm	Ⓡ	460 mm
Ⓕ	975 mm	Width, cut out in seat plate	205 mm
Ⓖ	620 mm	Depth, cut out in seat plate	264 mm
Ⓗ	160 mm	∅ Castors	127 mm (5")

Weights

Toilet Chair Cascata H720T4C	15.2 kg
Maximum user weight	130 kg

Materials

Ⓛ	Steel, powder coated	ⓂⓃⓅ	PVC
Ⓡ	PP		

All product components are corrosion resistant and do not contain natural rubber latex.

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1. Ziehen Sie den Hebel **©** nach oben und lösen Sie die Fußauflage, indem Sie sie anheben.
Invacare Limited