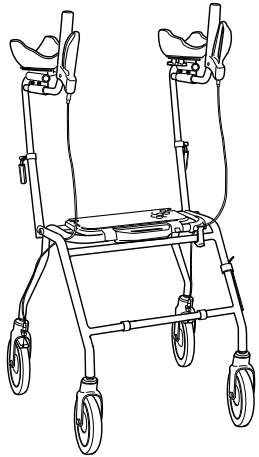
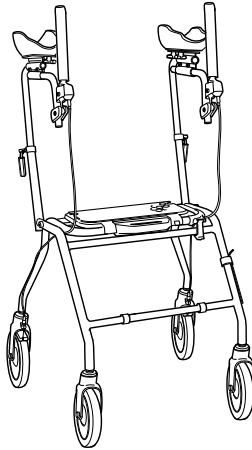
Dolomite® Alpha

Alpha Basic, Alpha Advanced

Alpha Basic

Alpha Advanced

en	Walking table User Manual	3
da	Ganghjælpemiddel Brugsanvisning	23
de	Gehhilfe Gebrauchsanweisung	41
fi	Kävelypöytä Käyttöohje	61
fr	Tablette de déambulation Manuel d'utilisation	81
it	Deambulatore Manuale d'uso	101
nl	Looptafel Gebruiksaanwijzing	123
no	Gåbord Bruksanvisning	143
sv	Gåbord Bruksanvisning	163

This manual **MUST** be given to the user of the product.
BEFORE using this product, this manual **MUST** be read and saved for future reference.

Dolomite

Yes, you can.®

©2020 Invacare Corporation

All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

Making Life's Experiences Possible is a registered trademark in the U.S.A.

Contents

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved
for future reference.

1 General	4
1.1 Introduction	4
1.2 Symbols in this Document	4
1.3 Compliance	4
1.4 Service Life	5
1.5 Warranty Information	5
1.6 Limitation of Liability	5
2 Safety	6
2.1 General Safety Information	6
2.2 Labels and Symbols on the Product	7
3 Product Overview	9
3.1 Product Description	9
3.2 Intended Use	9
3.3 Indications for Use	9
3.4 Scope of delivery	9
3.5 Accessories	9
4 Usage	10
4.1 Before Use	10
4.2 Assembling the walking aid	10
4.3 Folding and unfolding the walking aid	11
4.4 Adjusting the arm support	11
4.5 Adjusting the brake handle (Alpha Advanced)	13
4.6 Walking	13
4.7 Sitting Down on a Chair	14
4.8 Rising to a Standing Position	14
4.9 Sitting on the Walking Aid	15
4.10 Braking	15

5 Maintenance	17
5.1 General Maintenance Information	17
5.2 Inspection Checklist	17
5.3 Cleaning and Disinfection	17
5.3.1 General Safety Information	17
5.3.2 Cleaning Intervals	18
5.3.3 Cleaning Instructions	18
6 After Use	19
6.1 Disposal	19
6.2 Reconditioning	19
7 Technical Data	20
7.1 Dimensions and Weights	20
7.2 Material	21

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Note that there may be sections in this document, which are not relevant to your product, since this document applies to all available models (on the date of printing). If not otherwise stated, each section in this document refers to all models of the product.

The models and configurations available in your country can be found in the country-specific sales documents.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

For more information about the product, for example product safety notices and product recalls, contact your Invacare distributor. See addresses at the end of this document.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

1.2 Symbols in this Document

Symbols and signal words are used in this document and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

1.3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

1.4 Service Life

The expected service life of this product is five years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

1.5 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

1.6 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

2 Safety

2.1 General Safety Information

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

WARNING!

Risk of Serious Injury

- Incorrect use can lead to hazardous situations.
- Do not use on stairs or on escalators.
 - Do not stand on the product.
 - Be careful when walking on sloping or slippery ground or on uneven surfaces.
 - Avoid walking against obstacles, both forwards and backwards.
 - Do not let children play with the product.
 - Do not use the product to transport a person.
 - Make sure that no clothes, such as long skirts, can get stuck in the wheels.

CAUTION!

Risk of Injury

- Unbalanced or heavy load may lead to overturning.
- Do not hang any loads, such as bags, to the frame, the push handles or hand knobs.
 - Only carry loads with the designated equipment, like basket or tray. Do not exceed the maximum load for the equipment.
 - The total load, including the weight of the user and any additional load, must not exceed the maximum user weight.
 - Refer to chapter 7 *Technical Data*, page 20 for maximum user weight and load information.

CAUTION!
Risk of Injury

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalog or website for available options or contact your Invacare distributor. See addresses at the end of this document.

CAUTION!
Risk of First Degree Burns

Parts of the product may heat up, when exposed to high temperatures (above 41° C).

- Be careful if the product is exposed to high temperatures, e.g. direct sunlight.

IMPORTANT!

There is a risk of damage due to wear, e.g. corrosion.

- Do not use the product under water, e.g. in pools.
- Do not use the product in hot areas like saunas.
- Do not store the product in wet or humid environments.

2.2 Labels and Symbols on the Product

Identification label

- Ⓐ Product name
- Ⓑ Part number
- Ⓒ Date of manufacture
- Ⓓ Serial number
- Ⓔ Maximum user weight
- Ⓕ Maximum width
- Ⓖ Manufacturer address

Symbols

	European Conformity		Medical Device
	Manufacturer		Read user manual
	Date of manufacture		Serial number

Dolomite® Alpha

	Maximum width		Maximum user weight
	Indoor / Outdoor use		

3 Product Overview

3.1 Product Description

The four-wheeled walking aid is equipped with a central seat that can be folded up for storage and transportation. Front wheels are steering wheels. Rear wheels are provided with a brake system for slowing down and for locking the wheels. The walking aid is available in different sizes and can be equipped with a wide range of accessories.

3.2 Intended Use

The walking aid is intended to support during walking and may also serve as a resting seat.

The walking aid is suitable for indoor and outdoor use on even ground.

The intended user is an adult with a maximum user weight as stated on the product label and in chapter 7 *Technical Data*, page 20.

3.3 Indications for Use

Indications

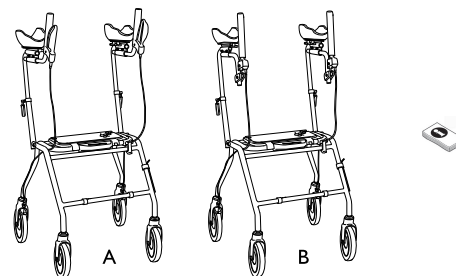
Persons with impaired muscle strength and/or balance, who need additional support whilst walking. The remaining walking and cognitive ability must be sufficient for safe use of a walking aid.

Contraindications

Persons with severe equilibrium disorders or functional perceptual disorders.

1576161-D

3.4 Scope of delivery

A: Alpha Basic

B: Alpha Advanced

- Walking aid, partly pre-assembled.
- User manual

For assembling see section 4.2 *Assembling the walking aid*, page 10.

Accessories, actually assembled to your walking aid, are depending on the configuration and might differ per region.

3.5 Accessories

Various accessories, like a basket / bag, are available for this product and can be found in the country-specific sales documents. Depending on the configuration accessories may be pre-assembled.

4 Usage

4.1 Before Use

CAUTION!

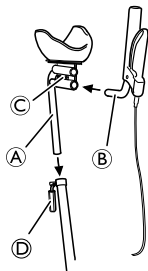
Risk of Injury or Damage to Property

- Before use, verify the general functionality of the product.
- Before use, check the product for any defect.
- Do not use a defective product.


Invacare presume a healthcare professional has made an individual assessment of the user to determine his risk level or therapy requirements. Providers and carers have assessed that the product gives the required support and that its adjustments are appropriate for the user. Carer and user have been appropriately trained by a healthcare professional on how to use the product correctly and safely.

4.2 Assembling the walking aid

Alpha Basic


Alpha Advanced

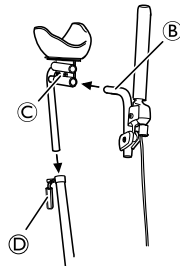1. Place the arm support tube (A) in the frame tube and secure with lever (D).
2. Place the handle bar (B) in the arm support tube and secure with the screws (C).
3. Repeat the procedure on the other side.

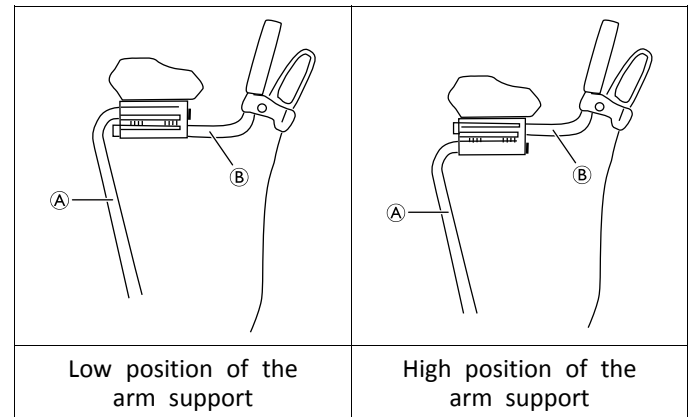The arm support tube and the handle bar can be placed in two different positions depending on the required height.


Low position:

1. Place the arm support tube (A) in the upper attachment position.
2. Place the handle bar (B) in the lower attachment position.


High position:

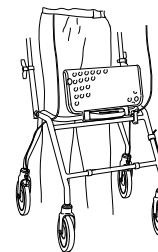1. Place the arm support tube (A) in the lower attachment position.
2. Place the handle bar (B) in the upper attachment position.

4.3 Folding and unfolding the walking aid

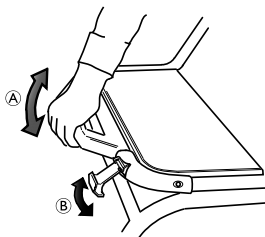Unfolding to operating position

1. Unfold the walking aid by pressing down the seating tube using the handle (A).

WARNING!

- Before using the product, always check that the safety latch (B) is in locked position indicated by a clear audible “click”.

Flip up the seat plate

The seat can be folded up to provide a larger walking space, and in order to increase accessibility.

Folding the walking aid

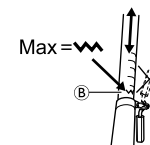
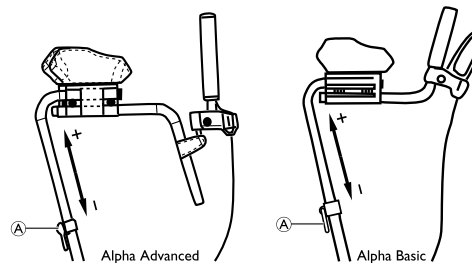
1. Follow the action above in reverse.

4.4 Adjusting the arm support

WARNING!

- All adjustments must be carried out by your specialist dealer.
- Before usage, always make sure that the levers (A) are pointing downwards and correctly tightened after height or angle adjustment.

Adjusting the height

1. Untighten the lever (A).
2. Adjust the arm support to a suitable height.
3. Tighten the lever.

WARNING! **Risk of falling**

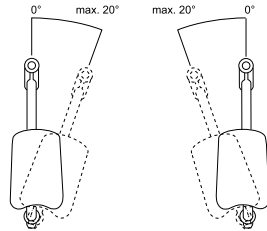
– Do not set the arm support above the highest position, indicated by the marking (B) on the tubes.

Adjusting the angle

1. Untighten the lever (A).
2. Adjust the angle of the handles to a suitable position.

Make sure that the handles are always in the allowed adjustment area (maximum 20° inwards).


Do not turn the handles outwards.


3. Tighten the lever.

Adjusting depth and inclination angle

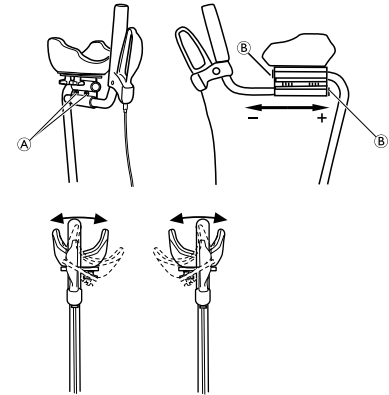Alpha Basic


Alpha Advanced

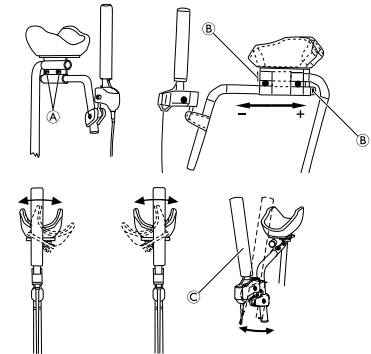1. Loosen the two screws (A) with the supplied Allen key.
2. Adjust the depth of the arm support to the desired position. Make sure that the black end plugs (B) are always visible.
3. Adjust the inclination angle of the arm support cup to the desired position.
4. Adjust the angle of the brake handle to the desired position.
5. Tighten the screws carefully.

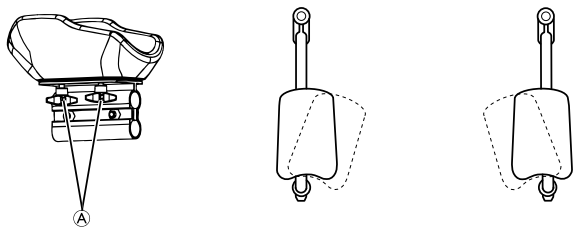
Rotating the arm support cups inwards

1. Loosen the two knobs (A).
2. Adjust the two arm support cups to appropriate width/angle.
3. Tighten the two knobs (A).

4.5 Adjusting the brake handle (Alpha Advanced)

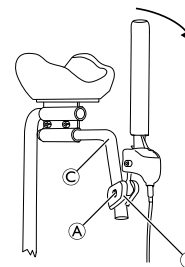The brake handle can be additionally adjusted for optional hand location and function direction.

1. Loosen the screw (A) with the supplied Allen key.
2. Adjust the fastening device (B) in height and sideways to the desired position.
3. Rotate the brake by means of the ball joint to the desired position.
4. Retighten the screw (A).

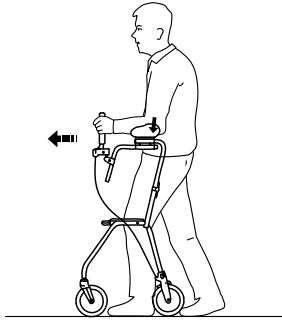
The brake can be set at a greater angle by twisting the pipe (C) in connection with the depth adjustment, → 4.4 *Adjusting the arm support*, page 11.

4.6 Walking


WARNING! **Risk of falling**

– Be careful if the wheels stop against an obstacle, e.g. a kerb.

1. Place the walking aid slightly in front of you and take a step towards it whilst using the hand grips for support.

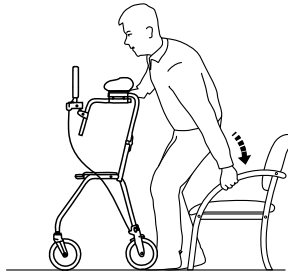4.7 Sitting Down on a Chair

1. Stand in front of the chair with the walking aid, the back of your legs should touch the chair.
2. Lock both hand brakes of the walking aid.
3. Place your hands on the arms of the chair, lean forward slightly and sit down carefully.

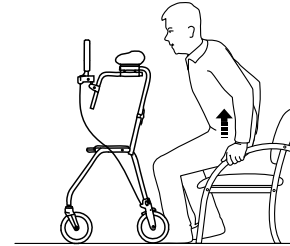
4.8 Rising to a Standing Position

WARNING!

Risk of tipping

- Do not use the walking aid to pull yourself up from a sitting position.

1. Place the walking aid in front of you and lock both hand brakes.
2. Use the armrests of the chair as support and rise carefully to a standing position.
3. Once stable in a standing position, transfer hands to the hand grips of the walking aid.

4.9 Sitting on the Walking Aid

WARNING!

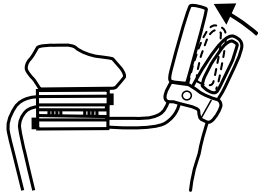
Risk of Falling

- Use the walking aid for stationary seating only.
- Before sitting down, lock both brakes in parking position and make sure the walking aid stands stable.
- Always sit on the walking aid from the rear side. Do not sit down from the front side.
- Do not sit on the walking aid on sloping ground.
- Do not maneuver the walking aid with your feet while sitting on it.


4.10 Braking

Braking during usage

Alpha Basic

Alpha Advance

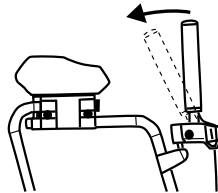1. Pull both brake handles (Alpha Basic) or brake levers (Alpha Advance) towards your body.

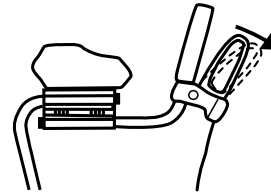
Applying the parking brakes


IMPORTANT!

- In parking position, always ensure that both brakes are applied.

Alpha Basic



Alpha Advance

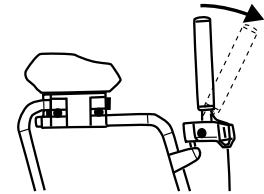1. Push both brake handles (Alpha Basic) or brake levers (Alpha Advance) away from your body. A clear audible “Click” indicates that the brake is in the right position.
2. To release the brakes, pull the brake handles (Alpha Basic) or brake levers (Alpha Advance) backwards.

Adjusting the brake effect

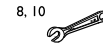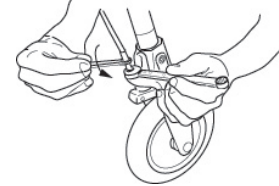For desirable braking strength, adjust the brake wire.

Suitable tools: 8 mm and 10 mm spanner (not included).

1. Loosen the locking nut and turn the hexagon nipple counter clockwise in order to increase the braking effect. The distance between the brake clamp and the wheel should be 2 mm.
2. Tighten the locking nut again to fixate the new adjustment.
3. Check the braking effect by engaging the parking brake and check that the wheel can't rotate.

WARNING!

– Make sure to tighten the lock nuts after adjustment.

5 Maintenance

5.1 General Maintenance Information

WARNING!
Risk of Injury

- Maintenance should be done at least once a year.
- Do not use a defective product.
- Do not attempt repairs yourself. For repairs contact your provider.

5.2 Inspection Checklist

To ensure good performance and safety, inspect your product regularly.

- Ensure that all screws and adjustments are properly tightened.
- Ensure that hand grips are tight.
If hand grips are loose do not use the walking aid. Instructions on how to change hand grips can be obtained from your provider or Invacare distributor.
- Verify brake operation.
If brake adjustment is required, see chapter 4 *Usage*, page 10.
- Inspect the frame and loaded parts for cracks.
- Inspect the wheels' tightness, wear and damage.
- Make sure that the wheels are free of hair, lint and other debris.

5.3 Cleaning and Disinfection

5.3.1 General Safety Information

CAUTION!
Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

IMPORTANT!

- Wrong fluids or methods can harm or damage the product.
- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
 - Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
 - Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
 - Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

5.3.2 Cleaning Intervals

IMPORTANT!

Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product

- regularly while it is in use,
- before and after any service procedure,
- when it has been in contact with any body fluids,
- before using it for a new user.

5.3.3 Cleaning Instructions

The product can be washed in a washing machine or by hand.

When using a high-pressure cleaner or an ordinary jet of water, do not aim the jet towards bearings, draining holes or labels.

The maximum cleaning temperature is 60 °C. Do not use a steam cleaner.

1. Clean the walking aid with a damp cloth and mild detergent.
2. Clean the wheels with a brush (no wire brush).
3. After washing, wipe the product dry.

Disinfection

- Wipe down all generally accessible surfaces with disinfectant.

Information on recommended disinfectants and methods can be found on <https://vah-online.de/en/for-users>.

6 After Use

6.1 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

6.2 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

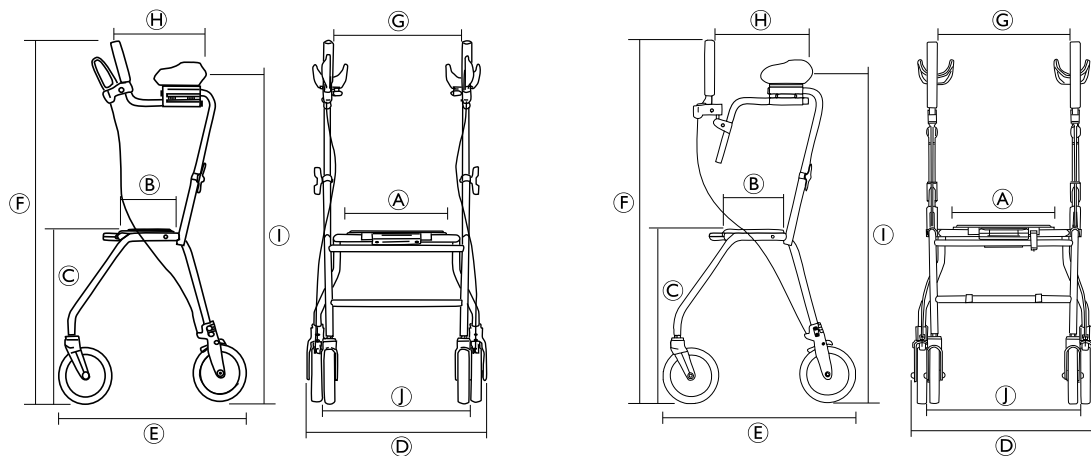
For detailed information, see *5 Maintenance, page 17* and the service manual for this product.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

7 Technical Data

7.1 Dimensions and Weights

	Alpha Basic	Alpha Advanced
Seat width (A)	360 mm	
Seat depth (B)	180 mm	
Seat height (C)	620 mm	
Total width (D)	650 mm	
Total depth (E)	700 mm	
Total height (Handle height) (F)	1075 – 1310 mm	1165 – 1400 mm
Distance between handles (G)	430 mm	

	Alpha Basic	Alpha Advanced
Distance between handle and lower arm support ④	290 – 330 mm	310 – 360 mm
Arm support height ①	950 – 1220 mm	
Distance between rear wheels ①	510 mm	
Front/rear wheel diameter	200 mm	

All dimensions are subject to certain tolerances

	Alpha Basic	Alpha Advanced
Weight ¹	11.3 kg	12.2 kg
Max. user weight ²	125 kg	

7.2 Material

Alpha Basic / Alpha Advanced	
Frame	Polyester (PES) powder coated steel
Handle bars	Aluminium
Handgrips	Closed cellular synthetic rubber
Wheels	Polypropylene (PP) with treads of Thermoplastic Elastomers (TPE)
Plastic parts	Material according to marking Polyamide (PA), Polypropylene (PP), Polyethylene (PE)
Screws	Steel, rust protected (zinc plated)

1. Total weight excluding equipment.

2. This weight limitation includes the weight of the user, whether standing or sitting, and any additional load, e.g. the content of the basket.

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Invacare Austria GmbH
Invacare Limited
Invacare Rea AB