





Yes, you can.®

Invacare® Dress Toileting High

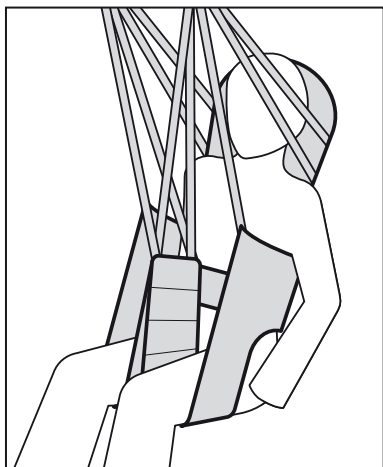


User's Manual (GB)
Brugermanual (DK)
Bruksanvisning (SE)
Brukerveiledning (NO)
Bedienungsanleitung (DE)
Gebruiksaanwijzing (NL)
Manual d'utilisation (FR)
Manual del usuario (ES)
Manuale d'uso (IT)
Manual de Utilização (PT)


Dress Toileting High

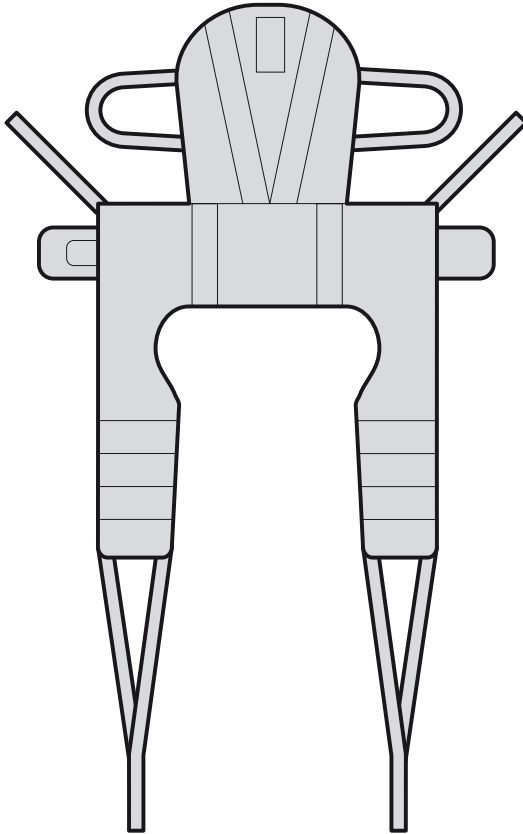


1.

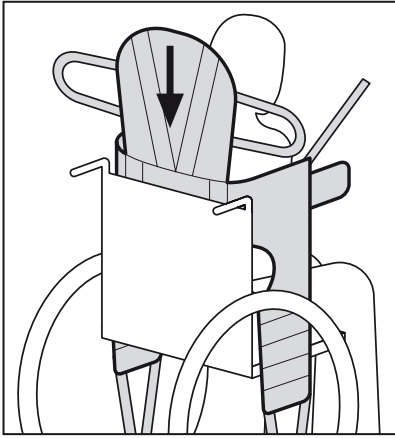


2.

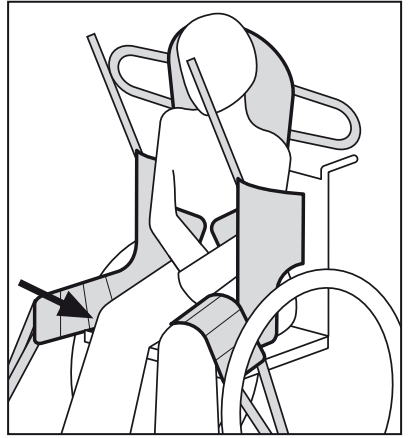

3.

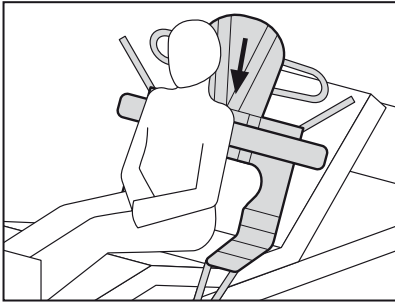


4.

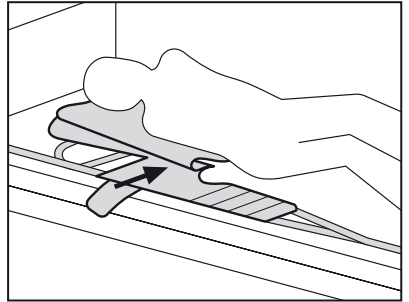

5.

6.

7.

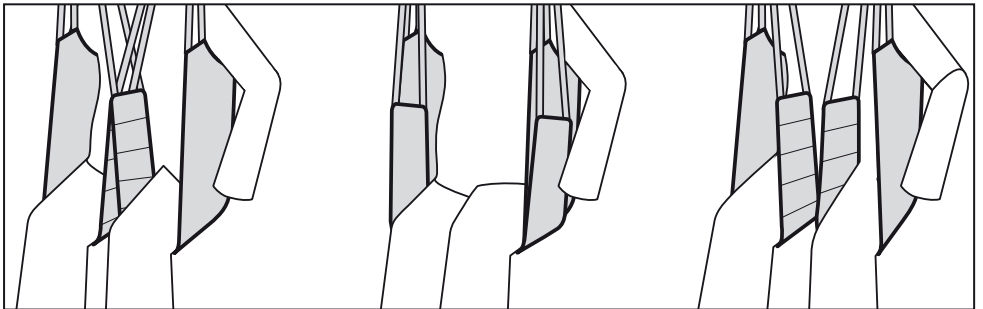

Quality Declaration

Congratulations with your new Invacare® sling from Invacare® EC-Høng A/S.

Your new sling is **CE**-marked in accordance with directive 93/42/EEC concerning medical devices.

Invacare® slings are developed and constructed with consideration for the user and others handling or assisting with lifting. Each sling type has been developed in accordance with the European Standard EN 10535.

Invacare® slings have been supervised and quality controlled throughout the entire production process, and the finished sling has been inspected by our finished goods control.

Please read the entire user's manual before using the sling.

Invacare® EC-Høng A/S is certified according to ISO 9001 and ISO 13485.

For and on behalf of Invacare EC-Høng A/S



Brian Mundeling
R & D Manager

Invacare® Dress Toileting High

USER'S MANUAL (GB)	2 - 3
BRUGERMANUAL (DK)	4 - 5
BRUKSANVISNING (SE)	6 - 7
BRUKERVEILEDNING (NO)	8 - 9
BEDIENUNGSANLEITUNG (DE)	10 - 11
GEBRUIKSAANWIJZING (NL)	12 - 13
MANUEL D'UTILISATION (FR)	14 - 15
MANUAL DEL USUARIO (ES)	16 - 17
MANUALE D'USO (IT)	18 - 19
MANUAL DE UTILIZAÇÃO (PT)	20 - 21

Dress Toileting High

Thank you for purchasing an Invacare® sling.

The following user guide must be read and adhered to, or the warranty may be void. Please check your sling on delivery – if the sling is not in perfect condition, please contact your supplier.

The Invacare® Toileting High sling combines quick and easy application with maximum personal access for dressing and hygiene. It is only suitable for clients with good trunk control but whom may need additional head support; however the sling can promote independence, as the arms remain free for function.

Sling application – Sitting in chair or wheelchair

Lean the client forward ensuring that he is well supported to reduce risk of falling and to promote feelings of security. Slide the sling down behind the back of the client in a central position with the aperture apex aligned with the waist - the padded upper edge must be positioned right under the arms. **Picture 1.**



The arms must be outside the sling.

Pass the leg support around the front of the chair to each side of the thigh and then under the thigh. **Picture 2.**

Sling application – Sitting in a bed

If the client is able to sit up from the bed or floor, this will ease the application. Lean the client forward ensuring that they are well supported to reduce risk of falling and to promote a feeling of security.

Slide the sling down behind the back of the client in a central position with the aperture apex aligned with the waist - the padded upper edge must be right under the arms.

Sit the client back against the sling. **Picture 3.**

Alternatively, if the client does not have sufficient trunk strength, roll the sling to mid-way and roll the client to one side to position the rolled up sling under the lifted side. Ensure the sling is placed centrally (green mark) along the spine. Roll the client to the other side, unroll the sling and roll the client back into position on the sling. **Picture 4.**

Bend the knees one at a time to apply the sling under the legs.



The arms must be outside the sling.

Application of the legs straps

Crossed legs straps: For maximum security, cross the legs straps. This will reduce the risk of forward pitching and also maintain a better alignment of the hips and lower limbs. **Picture 5.**

Looping the leg sections together: If the client feels this is more comfortable pass each individual leg strap under both legs.

Ensure that the client is not prone to spasm and there has been a risk assessment completed to address the possibility of forward pitching. **Picture 6.**

Uncrossed leg straps: Each leg strap can be applied to the spreader bar or lifting strap without first crossing over. This can be recommended when the client has abdominal sensitivity or requires intimate hygiene. **Picture 7.**

Note: It is crucial for the sitting position that the leg support is placed correctly. Clients with reduced hip control must have the leg supports placed close to the groin.

Attach loops to the spreader bar or lifting strap

Attach the loops to the spreader bar or lifting straps. Attach the shoulder- and head loops before the leg loops. The sling can be used at spreader bars with 2 or 4 attachment points. Please note, that for security and comfort reasons, the size (width) of the spreader bar must correspond with the size of the sling. Invacare recommend that a professional, prior to issuing lifting equipment, always carry out a risk assessment. If you are in doubt about the use of Invacare's slings with other manufacturer's equipment, contact Invacare® or your supplier for advice.

The position of the client in the sling can be adjusted by using the different adjustment straps. Shorter loops at the shoulders and head support (the blue straps) and longer at the legs (grey) produces a more vertical lift, which will assist with positioning into a chair or wheelchair. By lowering the loops at the shoulders/head (grey loops) and/or shorten the loops at the legs (blue loops), a more reclined position can be achieved, which is more suitable for a chair to bed transfer.



Use only CE marked spreader bars and hoists, preferably with a rated Safe Working Load equal to or higher than that of the sling. If the SWL is lower than of the sling, the SWL of the spreader bar or hoist is applicable.



Always check that all the straps are properly attached to the spreader bar or lifting straps before attempting the lift.



Invacare® recommend that a risk assessment is always carried out by a professional prior to issuing lifting equipment. Always choose the sling design and size according to the clients weight, size and physical ability.



Slings should be checked regularly for wear and damage* – NEVER use a damaged sling. (*In accordance with each country's regulations).



If you are in any doubt, please do not attempt to use the sling and seek the advice of a professional or your supplier.

Permissible max. load 200 kg.

Symbols:  = 200 kg   95   

Safe Working Load, consult the Users Manual, do not exceed 95°C washing temperature, do not bleach, do not tumble dry, do not iron.

All Invacare® slings are guaranteed for one year.



Yes, you can.®

Customer Sales and Service

Denmark

INVACARE A/S

Sdr. Ringvej 37
DK-2605 Brøndby
Phone: +45 36 90 00 00
Fax: +45 36 90 00 01
www.invacare.dk
denmark@invacare.com

Sweden & Finland

INVACARE AB

Fagerstagatan 9 / Box 66
S-163 91 Spånga
Phone: +46 8 761 70 90
Fax: +46 8 761 81 08
www.invacare.se
sweden@invacare.com

Norway & Iceland

INVACARE AS

Grensesvingen 9
Postbox 6230 Etterstad
N-0603 Oslo
Phone: +47 22 57 95 00
Fax: +47 22 57 95 01
www.invacare.no
norway@invacare.com

Spain

INVACARE S.A.

C/Areny S/N
Poligon Industrial de Celrà
E-17460 Celrà (Girona)
Phone: +34 972 49 32 00
Fax: +34 972 49 32 20
www.invacare.es
contactsp@invacare.com

Switzerland

MOBITEC REHAB AG

Benkenstrasse 260
CH-4108 Witterswil
Phone: +41 61 487 70 80
Fax: +41 61 487 70 81
switzerland@invacare.com

Invacare® EC-Høng A/S

Ident. no.: 1487380

Version: 02

Date: 06.2007

Belgium & Luxemburg

INVACARE N.V.

Autobaan 22
B-8210 Loppem, Brügge
Phone: +32 50 83 10 10
Fax: +32 50 83 10 11
www.invacare.be
belgium@invacare.com

Netherlands

INVACARE B.V.

Celsiusstraat 46
NL-6716 BZ Ede
Phone: +31 318 695 757
Fax: +31 318 695 758
www.invacare.com
nederland@invacare.com
csede@invacare.com

Germany

INVACARE AQUATEC GmbH

Alemannenstrasse 10
D-88316 Isny
Phone: +49 75 62 7 00 0
Fax: +49 75 62 7 00 66
www.invacare-aquatec.com
info@invacare-aquatec.com

Portugal

INVACARE Lda

Rua Estrada Velha 949
P-4465-784 Leça do Balio
Phone: +351 225 1059 46/47
Fax: +351 225 1057 39
www.invacare.pt
portugal@invacare.com

Australia

INVACARE Australia Pty Ltd

1 Lenton Place, North Rocks
NSW 2151
Phone: +61 2 8839 5333
Fax: +61 2 8839 5353
www.invacare.com.au
sales@invacare.com.au

3rd party certified
according to
EN ISO 9001
EN ISO 13485

France

INVACARE Poirier S.A.S

Route de St. Roch
F-37230 Fondettes
Phone: +33 2 47 62 64 66
Fax: +33 2 47 42 12 24
www.invacare.fr
contactfr@invacare.com

Italy

INVACARE MECC SAN S.R.L.

Via dei Pini 62
I-36016 Thiene (VI)
Phone: +39 0445 38 00 59
Fax: +39 0445 38 00 34
www.invacare.it
italia@invacare.com

United Kingdom & Ireland

INVACARE LTD

South Road
Bridgend Industrial Estate
UK-Bridgend, CF31 3PY
Phone: +44 1 656 664 321
Fax: +44 1 656 667 532
www.invacare.co.uk
uk@invacare.com
ireland@invacare.com

New Zealand

INVACARE NZ

4 Westfield Place,
Mt. Wellington
Auckland
Phone: +64 9 917 3939
Fax: +64 9 917 3957
www.invacare.co.nz
sales@invacare.co.nz

Austria

MOBITEC MOBILITÄTSHILFEN GmbH

Herzog Odilostrasse 101
A-5310 Mondsee
Phone: +43 6232 5535 0
Fax: +43 6232 5535 4
www.mobitec-austria.com
austria@invacare.com

Manufacturer:

INVACARE EC-Høng A/S

Østergade 3
DK-4270 Høng
www.invacarebeds.dk