

E2 HEADREST INTERFACE PLATE FOR ELITE E2 BACKS


The E2 Headrest Interface Plate is compatible with most industry standard headrest mounting hardware and is available in two models:

E2HI (Standard): The Standard E2 Headrest Interface Plate (A) is included with Elite E2S/E2D backs for **Back Heights** ranging from **14" (36cm)** up to **20" (51cm)**.

E2HDI (Heavy-Duty): The Heavy-Duty E2 Headrest Interface Plate offers an extra strong steel headrest mount for extreme tone or heavy use applications.

E2 HEADREST INTERFACE PLATE INSTALLATION

Item	Qty	Description
1	1	E2HI Standard E2 Headrest Interface Adapter
n/s	1	E2HDI Heavy-Duty E2 Headrest Interface Adapter
2	2	Headrest Nut Plate
3	2	M6 Nyloc Nut, BZ
4	2	M6 x 14 Socket Head Cap, Alloy Steel, BZ
5	2	M6-1.0 x 18mm Button Socket Cap, Alloy Steel, BZ

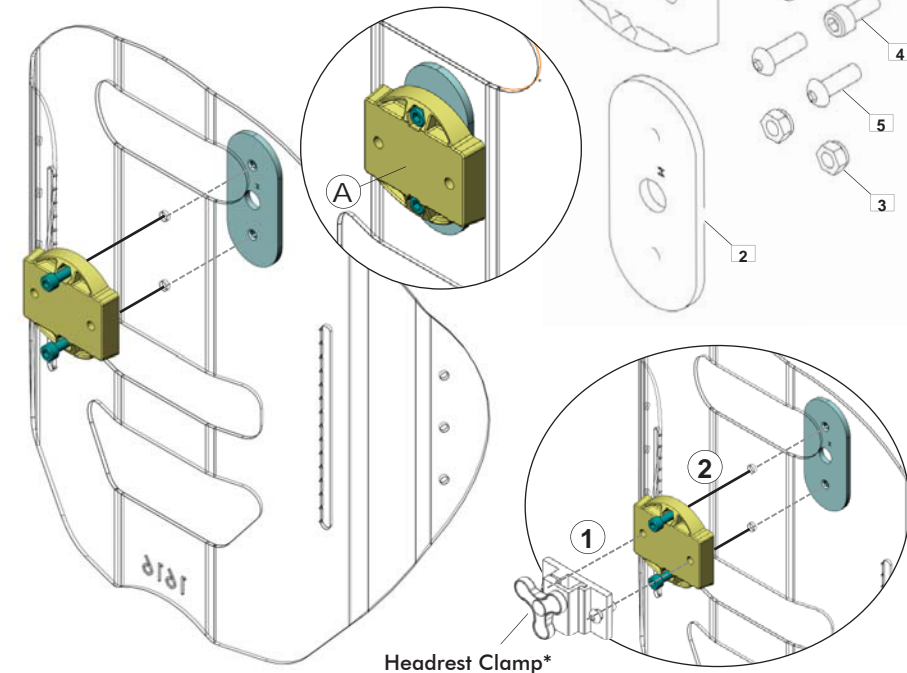

***NOTE:** The Headrest Clamp assembly (sold separately) must be installed onto the E2 Headrest Interface Plate **PRIOR** to installation onto the back shell

TRD0582 Rev A

Invacare Distributors:

Belgium & Luxembourg:
Invacare nv Autobaan 22
B-8210 Loppem
Tel: (32) (0)50 83 10 10
Fax: (32) (0)50 83 10 11
belgium@invacare.com
www.invacare.be

España:
Invacare SA
c/Areny s/n, Polígon Industrial
de Celrà
E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00
Fax: (34) (0)972 49 32 20
contactsp@invacare.com
www.invacare.es

Danmark:
Invacare A/S Sdr. Ringvej 37
DK-2605 Brøndby
Tel: (45) (0)36 90 00 00
Fax: (45) (0)36 90 00 01
denmark@invacare.com
www.invacare.dk

France:
Invacare Poirier SAS
Route de St Roch
F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66
Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com
www.invacare.fr

Portugal:
Invacare Lda
Rua Estrada Velha, 949
P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47
Fax: (351) (0)225 1057 39
portugal@invacare.com
www.invacare.pt

USA:
Invacare USA
Worldwide Headquarters
One Invacare Way
Elyria, Ohio,
USA 44036-2125
Tel: (1) 800 333 6900
Fax: (1) 877 619 7996
www.invacare.com

Deutschland:
Invacare GmbH
Alemannenstraße 10
D-88316 Isny
Tel: (49) (0)7562 700 0
Fax: (49) (0)7562 700 66
kontakt@invacare.com
www.invacare.de

Ireland:
Invacare Ireland Ltd,
Unit 5 Seatown Business Campus
Seatown Road, Swords,
County Dublin
Tel: (353) 1 810 7084
Fax: (353) 1 810 7085
ireland@invacare.com
www.invacare.ie

Eastern Europe, Middle East & CIS:
Invacare EU Export Kleiststraße 49
D-32457 Porta Westfalica /
Germany
Tel: (49) (0)57 31 754 540
Fax: (49) (0)57 31 754 541
webinfo-eu-export@invacare.com
www.invacare-eu-export.com

Italia:
Invacare Mecc San s.r.l.,
Via dei Pini 62,
I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59
Fax: (39) 0445 38 00 34
italia@invacare.com
www.invacare.it

United Kingdom:
Invacare Limited
Pencoed Technology Park,
Pencoed
Bridgend CF35 5AQ
Tel: (44) (0) 1656 776 222
Fax: (44) (0) 1656 776 220
uk@invacare.com
www.invacare.co.uk

Canada:
Invacare Canada
570 Matheson Blvd. E. Unit 8
Mississauga, Ontario
Canada L4Z 4G4
Tel: (800) 668 5324
Fax: (905) 890 5244
www.invacare.ca

Nederland:
Invacare BV Galvanistraat 14-3
NL-6716 AE Ede
Tel: (31) (0)318 695 757
Fax: (31) (0)318 695 758
nederland@invacare.com
www.invacare.nl

Suomi:
Camp Mobility
Patamäenkatu 5,
33900 Tampere
Puhelin 09-35076310
info@campmobility.fi
www.campmobility.fi

Norge:
Invacare AS
Grensesvingen 9, Postboks 6230,
Etterstad
N-0603 Oslo
Tel: (47) (0)22 57 95 00
Fax: (47) (0)22 57 95 01
norway@invacare.com
www.invacare.no

Sverige:
Invacare AB Fagerstagatan 9
S-163 53 Spånå
Tel: (46) (0)8 761 70 90
Fax: (46) (0)8 761 81 08
sweden@invacare.com
www.invacare.se

Österreich:
Invacare Austria GmbH Herzog Odi-
lostrasse 101
A-5310 Mondsee
Tel: (43) 6232 5535 0
Fax: (43) 6232 5535 4
info-austria@invacare.com
www.invacare.at

Schweiz / Suisse / Svizzera:
Invacare AG Benkenstrasse 260
CH-4108 Witterswil
Tel: (41) (0)61 487 70 80
Fax: (41) (0)61 487 70 81
switzerland@invacare.com
www.invacare.ch



Motion Concepts
84 Citation Drive, Unit 1
Concord, Ontario L4K 3C1
Canada



Advena Ltd.
Tower Business Cebter, 2nd Flr
Tower Street, Swatar BKR 4103
Malta



E2 HEADREST INTERFACE PLATE FOR ELITE E2 BACKS


The E2 Headrest Interface Plate is compatible with most industry standard headrest mounting hardware and is available in two models:

E2HI (Standard): The Standard E2 Headrest Interface Plate (A) is included with Elite E2S/E2D backs for **Back Heights** ranging from **14" (36cm)** up to **20" (51cm)**.

E2HDI (Heavy-Duty): The Heavy-Duty E2 Headrest Interface Plate offers an extra strong steel headrest mount for extreme tone or heavy use applications.

E2 HEADREST INTERFACE PLATE INSTALLATION

Item	Qty	Description
1	1	E2HI Standard E2 Headrest Interface Adapter
n/s	1	E2HDI Heavy-Duty E2 Headrest Interface Adapter
2	2	Headrest Nut Plate
3	2	M6 Nyloc Nut, BZ
4	2	M6 x 14 Socket Head Cap, Alloy Steel, BZ
5	2	M6-1.0 x 18mm Button Socket Cap, Alloy Steel, BZ

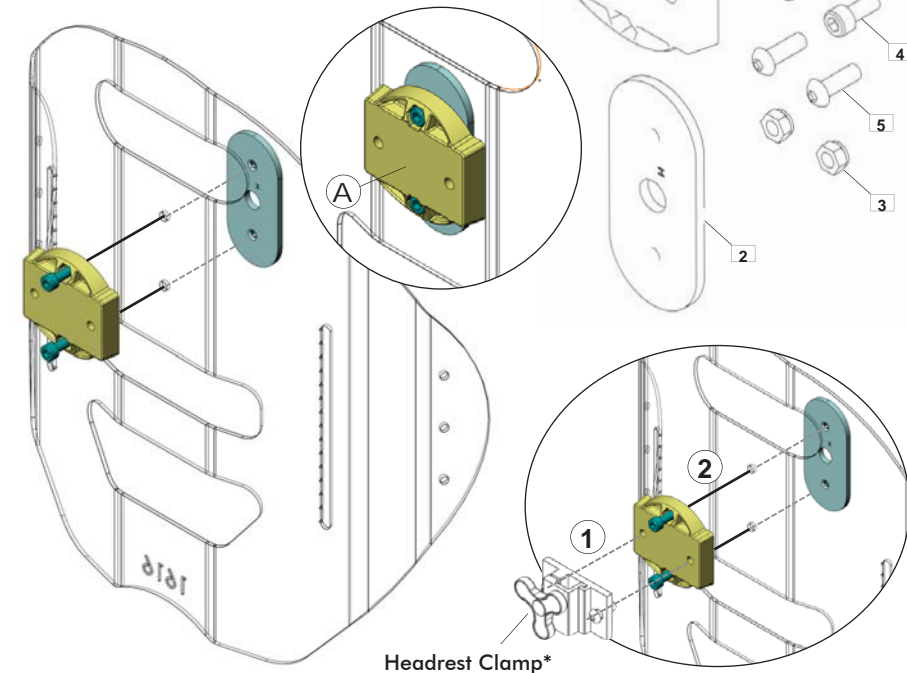

***NOTE:** The Headrest Clamp assembly (sold separately) must be installed onto the E2 Headrest Interface Plate **PRIOR** to installation onto the back shell

TRD0582 Rev A

Invacare Distributors:

Belgium & Luxembourg:
Invacare nv Autobaan 22
B-8210 Loppem
Tel: (32) (0)50 83 10 10
Fax: (32) (0)50 83 10 11
belgium@invacare.com
www.invacare.be

España:
Invacare SA
c/Areny s/n, Polígon Industrial
de Celrà
E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00
Fax: (34) (0)972 49 32 20
contactsp@invacare.com
www.invacare.es

Danmark:
Invacare A/S Sdr. Ringvej 37
DK-2605 Brøndby
Tel: (45) (0)36 90 00 00
Fax: (45) (0)36 90 00 01
denmark@invacare.com
www.invacare.dk

France:
Invacare Poirier SAS
Route de St Roch
F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66
Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com
www.invacare.fr

Portugal:
Invacare Lda
Rua Estrada Velha, 949
P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47
Fax: (351) (0)225 1057 39
portugal@invacare.com
www.invacare.pt

USA:
Invacare USA
Worldwide Headquarters
One Invacare Way
Elyria, Ohio,
USA 44036-2125
Tel: (1) 800 333 6900
Fax: (1) 877 619 7996
www.invacare.com

Deutschland:
Invacare GmbH
Alemannenstraße 10
D-88316 Isny
Tel: (49) (0)7562 700 0
Fax: (49) (0)7562 700 66
kontakt@invacare.com
www.invacare.de

Ireland:
Invacare Ireland Ltd,
Unit 5 Seatown Business Campus
Seatown Road, Swords,
County Dublin
Tel: (353) 1 810 7084
Fax: (353) 1 810 7085
ireland@invacare.com
www.invacare.ie

Eastern Europe, Middle East & CIS:
Invacare EU Export Kleiststraße 49
D-32457 Porta Westfalica /
Germany
Tel: (49) (0)57 31 754 540
Fax: (49) (0)57 31 754 541
webinfo-eu-export@invacare.com
www.invacare-eu-export.com

Italia:
Invacare Mecc San s.r.l.,
Via dei Pini 62,
I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59
Fax: (39) 0445 38 00 34
italia@invacare.com
www.invacare.it

United Kingdom:
Invacare Limited
Pencoed Technology Park,
Pencoed
Bridgend CF35 5AQ
Tel: (44) (0) 1656 776 222
Fax: (44) (0) 1656 776 220
uk@invacare.com
www.invacare.co.uk

Canada:
Invacare Canada
570 Matheson Blvd. E. Unit 8
Mississauga, Ontario
Canada L4Z 4G4
Tel: (800) 668 5324
Fax: (905) 890 5244
www.invacare.ca

Nederland:
Invacare BV Galvanistraat 14-3
NL-6716 AE Ede
Tel: (31) (0)318 695 757
Fax: (31) (0)318 695 758
nederland@invacare.com
www.invacare.nl

Suomi:
Camp Mobility
Patamäenkatu 5,
33900 Tampere
Puhelin 09-35076310
info@campmobility.fi
www.campmobility.fi

Norge:
Invacare AS
Grensesvingen 9, Postboks 6230,
Etterstad
N-0603 Oslo
Tel: (47) (0)22 57 95 00
Fax: (47) (0)22 57 95 01
norway@invacare.com
www.invacare.no

Sverige:
Invacare AB Fagerstagatan 9
S-163 53 Spånå
Tel: (46) (0)8 761 70 90
Fax: (46) (0)8 761 81 08
sweden@invacare.com
www.invacare.se

Österreich:
Invacare Austria GmbH Herzog Odi-
lostrasse 101
A-5310 Mondsee
Tel: (43) 6232 5535 0
Fax: (43) 6232 5535 4
info-austria@invacare.com
www.invacare.at

Schweiz / Suisse / Svizzera:
Invacare AG Benkenstrasse 260
CH-4108 Witterswil
Tel: (41) (0)61 487 70 80
Fax: (41) (0)61 487 70 81
switzerland@invacare.com
www.invacare.ch



Motion Concepts
84 Citation Drive, Unit 1
Concord, Ontario L4K 3C1
Canada



Advena Ltd.
Tower Business Cebter, 2nd Flr
Tower Street, Swatar BKR 4103
Malta

