Cushion Rigidizer

en	Seating System User Manual	1
de	Sitzsystem Gebrauchsanweisung	4
fr	Système d'assise Manuel d'utilisation	7
nl	Zittingsysteem Gebruiksaanwijzing	10
it	Sistema di seduta Manuale d'uso	12
es	Sistema de asiento Manual del usuario	15
pt	Sistema de assento Manual de utilização	18

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

Symbols in this Document

Symbols and signal words are used in this document and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING
Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION
Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

NOTICE
Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations
Gives useful tips, recommendations and information for efficient, trouble-free use.

Other Symbols

(Not applicable for all manuals)

UKRP UK Responsible Person
Indicates if a product is not manufactured in the UK.

Triman
Indicates recycling and sorting rules (only relevant for France).

1.2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

1.3 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

2 Safety

2.1 General Safety Information

WARNING!

Risk of Death, Injury or Damage

Improper use of the product may cause death, injury or damage. This section contains important information for the safe operation and use of this product.

- DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals, service manuals or instruction sheets supplied with this product or optional equipment.
- If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, provider or technical personnel before attempting to use this equipment.
- Check all external components and carton for damage. In case of damage, or if the product is not working correctly, contact a technician or Invacare for repair.

WARNING!

Risk of Serious Injury or Damage

Risk of injury or damage from improper installation of the **Cushion Rigidizer**.

- The procedures in this manual **MUST** be performed by a qualified technician.
- Movement of the **Cushion Rigidizer** within the cushion may occur causing discomfort to the user if not properly installed.
- Do **NOT** use this accessory on wheelchairs with a rigid seat pan.
- Use this accessory **ONLY** on wheelchairs with either nylon or sling seat upholsteries.

WARNING!

Risk of Serious Injury or Damage

Use of non-Invacare accessories may result in serious injury or damage.

- Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are **NOT** recommended for use with Invacare products.
- Do **NOT** use non-Invacare accessories.
- To obtain Invacare accessories, contact Invacare by phone or at www.invacare.eu.com.

WARNING!

Risk of Serious Injury or Damage

Attaching hardware that is loosely secured could cause loss of stability resulting in serious injury or damage.

- After **ANY** adjustments, repair or service and before use, make sure that all attaching hardware is tightened securely.

WARNING!

Risk of Serious Injury or Damage

Risk of injury or damage from exceeding the weight limit of the wheelchair or the **Cushion Rigidizer**.

- Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are **NOT** recommended for use with Invacare products.
- Use of the **Cushion Rigidizer** does **NOT** change the weight limitation of the wheelchair.
- The **Cushion Rigidizer** has a weight limitation of **136 kg (300 lbs)**.

NOTICE!

- The information contained in this document is subject to change without notice.


3 Product Overview

3.1 Product Description

The Invacare **Cushion Rigidizer** fits between the seat rails for level support and to reduce the hammock effect of the seat upholstery. It securely attaches to the bottom of the cushion inside the cushion cover.

3.2 Scope of Delivery

The following components are included within the scope of delivery: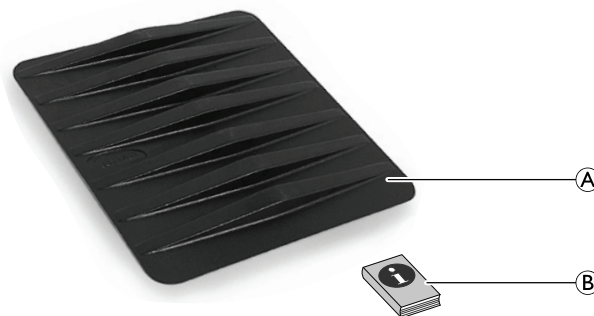

(A)	Cushion Rigidizer Material — formed ABS plastic.
(B)	User Manual


4 Setup

4.1 Installing the Cushion Rigidizer


To change the depth of the **Cushion Rigidizer**, see 4.2 *Customizing the Cushion Rigidizer*, page 3.


1. Remove the seat cushion assembly from chair.
2. Open seat cover (A) and remove the foam cushion (B) from the seat cover.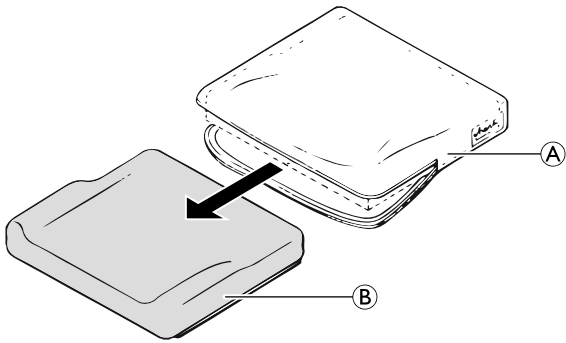

i Do **NOT** remove backing from double sided tape (C) until you have marked the placement area on the foam cushion. Be sure of the placement area as the **Cushion Rigidizer** will not re-adhere securely if installed and removed.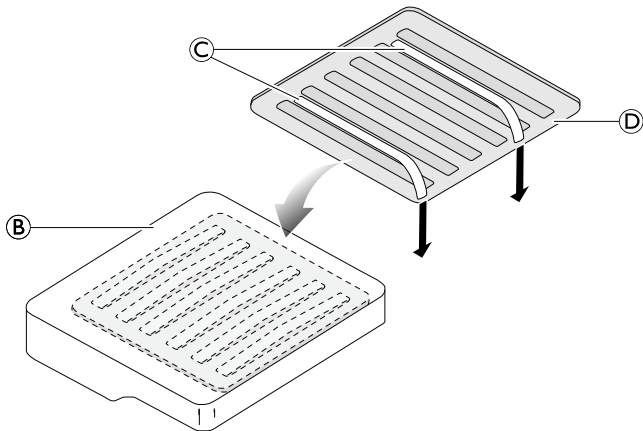

3. Position the bottom (underside) of the **Cushion Rigidizer** (D) onto the bottom of the foam cushion (B) to the extreme rear while centering it on the left and right sides.
4. Outline the foam cushion with a marker, pen or pencil using the **Cushion Rigidizer** as your template to mark the placement area.
5. After marking the placement area, remove the double-sided tape backing (C) from the **Cushion Rigidizer** and install it onto the bottom (underside) of the foam cushion.
6. Press down firmly to adhere the **Cushion Rigidizer** to the foam cushion.
7. Re-install foam cushion (B) into the seat cover (A) and secure seat cover to chair with the hook and loop backing.

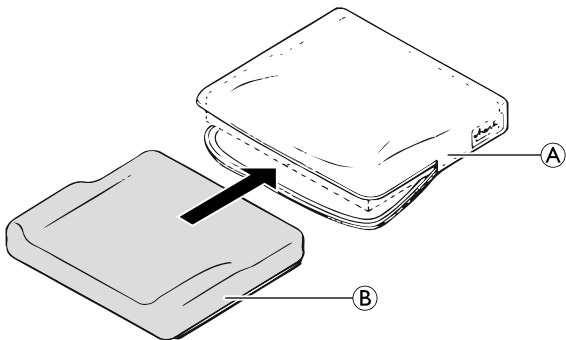4.2 Customizing the Cushion Rigidizer

The **Cushion Rigidizer** is available in various widths and depths, and can be trimmed to fit shorter seat depths.

i Presented measurements are in “Width (W) x Depth (D)” format.


38 cm (15 inch) deep **Cushion Rigidizers** (A) can be trimmed **5 cm (2 inches)** (B) to fit **36 cm (14 inch)** cushion depth, as shown.

CAUTION!

Risk of damage to the cushion cover

– When trimming the **Cushion Rigidizer**, ensure the cut edges are smooth to reduce wear on the inner side of the cushion cover.

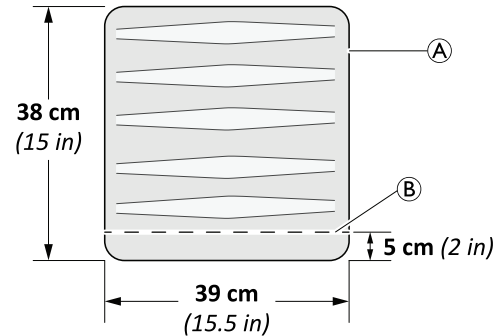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- Nadat het plaatsingsvlak is afgetekend, kunt u de beschermstrook van de dubbelzijdige tape © van de **kussenversteviging** verwijderen en de versteviging op de onderkant van het schuimkussen plaatsen.
Invacare Poirier SAS
Invacare UK Operations Limited